Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

Alevo

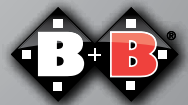
DE

EN

ES

FR

Enjoy mobility.

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www.bischoff-bischoff.com

Index

1. Introduction	24	5. Use	34
1.1. Signs and symbols	24	6. Technical data	35
2. Important safety instructions	25	7. Maintenance tips for users	37
2.1. General safety instructions	25	7.1. Cleaning and disinfection	37
2.2. Indications / Contraindications	25	7.2. Rollator check by the user	37
2.3. Purpose	26	7.2.1. Brake Inspection	37
2.4. Declaration of conformity	26	7.2.2. Checking the tyres	37
2.5. Responsibility	26	8. Maintenance tips for retailers	38
2.6. Service life	27	9. Storage/shipping	39
3. Description of product and contents	27	10. Transferring the rollator	39
3.1. Inspection on delivery	27	11. Disposal	39
3.2. Contents	27	12. Warranty	40
3.3. Type plate and serial number	28		
3.4. Summary	29		
4. Assembly/adjustments	30		
4.1. Unfolding	30		
4.2. Folding	30		
4.3. Attaching the bag	30		
4.4. Adjusting the push handles	30		
4.5. Adjusting the back strap	31		
4.6. Adjusting / activating the brakes	32		
4.7. Wheels and tyres	33		
4.8. Tipping Aid	33		
4.9. Cane holder (optional)	33		

1. Introduction

Dear user,

By purchasing a B+B rollator, you are now the owner of a top-quality German product that is setting new standards in terms of its variability. This instruction manual is intended to help you become more familiar with the operation and use of the rollator.

The Alevo rollator is offered in seat heights 585 mm and 510 mm.

Please read and follow the instruction manual carefully before using your new rollator for the first time. The instruction manual is an integral and necessary component of the product. Keep the instruction manual where you can refer to it, and if you transfer the rollator to another user, please provide the manual to the new owner.

For visually impaired users, this document is available as a PDF file at our website www.bischoff-bischoff.com.

Any repairs and adjustments require special technical training and may therefore only be carried out by authorized B+B dealers.

1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

2. Important safety instructions

2.1. General safety instructions

- Max. user weight: 130kg.
- Pinch risk! Be careful when folding up and unfolding the rollator. Make sure that clothing and body parts are not pinched between the folding parts of the rollator.
- Always use both brakes at the same time!
- The bag can carry a maximum weight of 5kg.
- Do not use aggressive or caustic detergents to clean the rollator!
- Check that the rollator is fully intact each time before using it. Check that the brake is working. Check that the back strap is fixated properly.
- The surfaces of the rollator can heat up when exposed to direct sunlight. There is a risk of injury.
- Reporting incidents. The operator or user must immediately report
 1. any malfunction
 2. any change to the features or performance as well as
 3. any inaccuracies in the marking or operating instructions of a medical device, which has or could have led to death or to a serious deterioration of the health of a patient, an employee or a third party, to the Federal Institute for Drugs and Medical Devices. This institute will immediately forward the notification to the operator's competent authority and also inform the manufacturer and the manufacturer's competent authority.

2.2. Indications / Contraindications

- The rollator supports mobility-impaired users with walking, e.g. at home, when shopping or walking outdoors.
- The seat area allows the user to rest during longer periods of use.
- Only use the rollator on a level and firm surface.
- The product may not be used as an aid for climbing or for transportation of heavy loads.

- Reporting incidents. The operator or user must immediately report
 1. any malfunction
 2. any change to the features or performance as well as
 3. any inaccuracies in the marking or operating instructions of a medical device, which has or could have led to death or to a serious deterioration of the health of a patient, an employee or a third party, to the Federal Institute for Drugs and Medical Devices. This institute will immediately forward the notification to the operator's competent authority and also inform the manufacturer and the manufacturer's competent authority.

2.3. Purpose

The rollator is intended to support mobility of persons with an impaired ability to walk, in accordance with the above-mentioned indications.

Please note:

We are only able to issue a guarantee if the product is used in accordance with the specified conditions and for the specified purpose.

2.4. Declaration of conformity

B+B, as manufacturer and solely responsible party, hereby declares that the rollator Alevo complies with the requirements of the EC Directive 93/42.

2.5. Responsibility

We are only able to provide a warranty if the product is used in accordance with the specified conditions and for the intended purposes. If modifications, repairs or maintenance works are made, please make sure only authorised persons perform these works in accordance to the instructions of B+B.

2.6. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time the rollator can be used continuously, provided it is in a safe condition.

3. Description of product and contents

3.1. Inspection on delivery

All B+B rollators are subject to a proper final inspection in our house and receive the CE mark.

The rollator is supplied in a special box. After unpacking save the box if possible. You may need it later for storage or to return the product.

Assembly, adjustment to your body size and first instructions are taken care of by your qualified medical supply dealer.

If delivered by rail or a forwarding agent, the product is to be checked with regard to transportation damage at once in the presence of the shipper.

Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service (Tel.: +49 700 6000 7070).

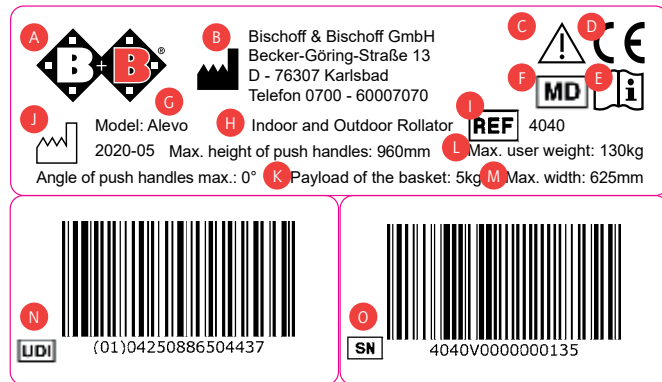
3.2. Contents

After receiving the goods, please check the contents for completeness right away. The content consists of:

- Outer package
- Rollator pre-assembled
- Instruction manual

3.3. Type plate and serial number


The type plate (Fig. 1) and serial number are located on the bottom of the frame.


- A** Manufacturer's logo
- B** Manufacturer's information
- C** Caution! - Follow instructions for use
- D** CE mark
- E** Important! - Follow instructions for use
- F** Medical device
- G** Model designation
- H** Application class
- I** Model number
- J** Date of manufacture
- K** Carrying capacity of bag
- L** Maximum load capacity
- M** Required dimensions
- N** UDI item number
- O** Serial number

3.4. Summary

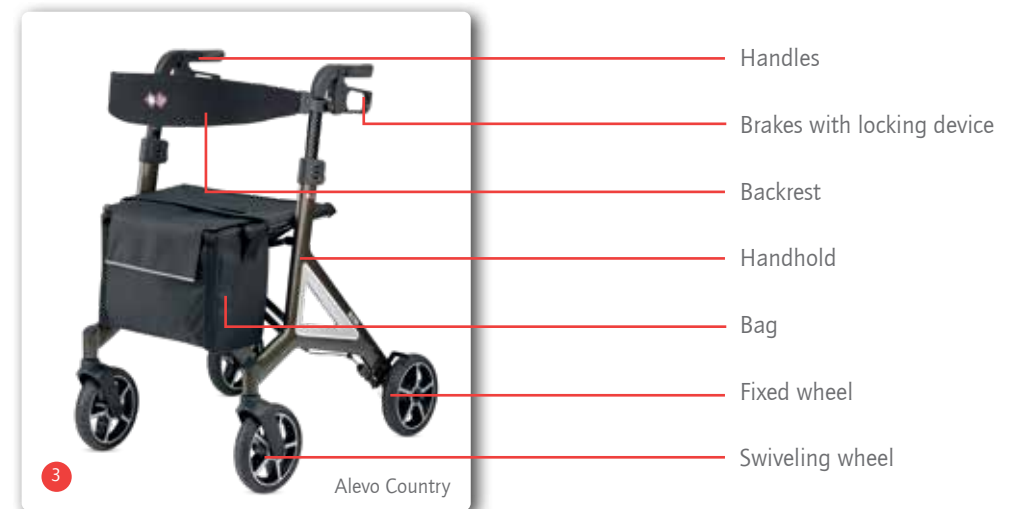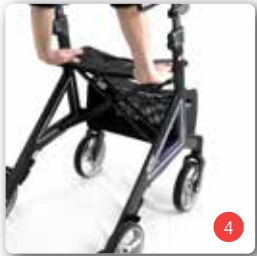Product overview Alevo (Fig. 2, Fig. 3)


4. Assembly/adjustments


4.1. Unfolding

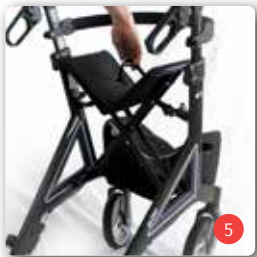- Position the rollator in the travel direction.
- Release the locking brakes (see sec. 4.6.).
- Press the frame tubes of the seat downward to unfold the rollator (Fig. 4). Make sure the tubes are aligned properly and snap into place.

4.2. Folding

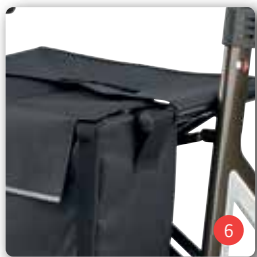- Activate the locking brakes.
- Hold the rollator firmly with one hand on the handle. With your other hand pull the grip on the seat upward (Fig. 5).
- When doing so you must overcome the resistance of the locking device.

4.3. Attaching the bag

- The rollator is unfolded. Activate the locking brakes.
- Position yourself against the travel direction in front of the rollator.
- Pull the straps of the bag over the two plastic handles (Fig. 6).

4.4. Adjusting the push handles

- The rollator is unfolded. Activate the locking brakes.
- Hold the push handle firmly with one hand. With the other hand, activate the locking device (Fig. 7).
- Move the push handle to the desired height and let go of the locking device.
- Now move the push handle up and down slightly to ensure that it snaps audibly into place.
- Repeat this procedure on the other side.

- On the inside of the push handles you'll find a scale (Fig. 8), which you can use to check the height of the handle. Make sure that both the number and the marking line is visible.
- Check to make sure that both the push handles are at the same height.


Setting the push handle height according to body height.

Level 0	– 1.59 m (1590 mm) or taller
Level 1	– 1.65 m (1600 mm) or taller
Level 2	– 1.70 m (1700 mm) or taller
Level 3	– 1.75 m (1750 mm) or taller
Level 4	– 1.80 m (1800 mm) or taller
Level 5	– 1.85 m (1850 mm) or taller
Level 6	– 1.90 m (1900 mm) or taller

Use these figures as guidelines; they may vary in practice.

4.5. Adjusting the back strap

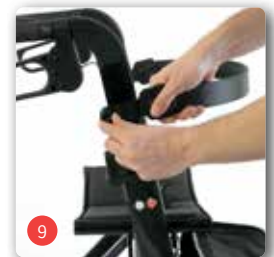- To adjust the height of the back strap, the hand screws on the frame (Fig. 9) must be loosened.
- Move the fastener for the back strap to the desired height and re-tighten the screws.
- The fasteners for the back strap should be at the same height on both sides.

4.6. Adjusting / activating the brakes

Service brake

To activate the service brake, pull both handbrake levers upward on the push handles upward (Fig. 10). Service and locking brake act on the fixed wheels.

Locking brake

- Press down on both brake levers to activate the locking device (Fig. 11).
- To release the locking brake pull both brake levers back into middle position.

Adjusting the brakes

The brake is set properly, if the brake pins do not touch the wheel and the locking brake can be activated easily.

- You can adjust the brake for harder or softer action by pressing in and then rotating the protective cap (Fig. 12).
- If the wheel can still be moved despite the activated brake, you need to adjust the brake for harder action. To do so, turn the protective cap (Fig. 12) on the brake mechanism in a counter-clockwise direction.
- If the brake rubs, you need to adjust the brake for softer action. To do so, turn the protective cap (Fig. 12) on the brake mechanism in a clockwise direction.

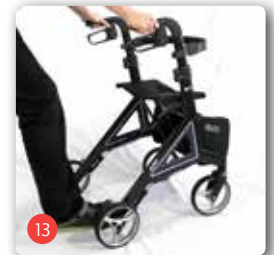
4.7. Wheels and tyres

Your rollator is equipped with flat-proof wheels (see sec. 6).

4.8. Tipping Aid

The tipping aid is intended to make it easier for users to overcome curbs and small obstacles.

Step on the tipping aid with your foot (Fig. 13) and at the same time pull on the push handles. The front wheels lift slightly off the ground and with a little force you can push the rollator slightly past the obstacle.

Caution! Make sure the rollator is positioned on level ground when using the tipping aid.

4.9. Cane holder (optional)

The optional cane holder can be attached with only a few simple steps. You can place both parts of the cane holder easily on the frame of the rollator (Fig. 14).

5. Use

Walking

Make your first attempts to walk with the rollator on a level, firm surface. Familiarise yourself with the function of the brakes. Let someone accompany you during this initial period.

For an anatomically correct gait, the push handles must be set at proper height (see sec. 4.4.).

- Place yourself in the rollator so that your ankles are at the height of the rear wheels.
- Stand with good posture.
- Grasp the push handles with both hands and your arms bent at a slight angle.

When walking with the rollator try to stay as close as possible with the body to the push handles. This increases the stability and safety against tipping over; in addition, avoid a crooked back. Try to generally walk in the rollator and not behind it. Take small steps and do not push the rollator too far ahead.

You can determine the change of direction and speed by pushing the rollator. Always use the brakes to slow down your movement. The rollator could otherwise turn with the unbraked wheel and the braked wheel, and could fall over.

Go around obstacles whenever possible. Try to use ramps or take lifts. If this is not possible, the rollator must be carried over the obstacle. Find a second person to help you do this, if possible.

Seating area

The rollator is also designed to be used as a seat by the user for short breaks. It is therefore equipped with a seating area and a back support. You must absolutely activate the locking brakes before sitting down. The rollator could roll away and you could fall.

Lifting points

You can lift the rollator on the grips or frame sections, in order to carry it over obstacles or steps.

6. Technical data

Dimensions and weights

Type:	Lightweight rollator
Max. user weight:	130 kg
Max. load of bag:	5 kg
Colour:	platinum / night blue / silver***, mocha***, aubergine***
Height:	810 mm - 960 mm
Length:	690 mm
Width:	625 mm
Width between handles:	460 mm
Width of handles:	30 mm (support surface 50 mm)
Seat height:	510 mm (*, **), 585 mm
Seat width:	445 mm
Folded:	265 mm
Reversing width:	850 mm

	Alevo Carbon	Alevo Alu 510 mm	Alevo Alu 580 mm	Alevo Country 510 mm	Alevo Country 580 mm
Total weight approx.	7 kg	7,13 kg	7,6 kg	7,3 kg	7,8 kg
Weight without bag and backstrap approx..	6,2 kg	6,53 kg	7 kg	6,53 kg	7 kg

Tyres

Rear wheels:	200 x 35 mm, 190 x 45 mm**
Steered wheels:	200 x 35 mm, 190 x 45 mm**
Tyres:	PU, Rubber**

Material

Frame:	Aluminum, Carbon***
Handles:	PP / TPE-SEPS
Seat:	Nylon / PU + foam***

Operating conditions

Temperature:	0 to 50°C
Air humidity:	20 - 80%

Storage conditions

Temperature:	-20 to + 60°C
Air humidity:	45 - 60%

- Including bag
- Optional cane holder
- Anatomically shaped handles
- Combined operating and locking brake
- Robust aluminum design, corrosion-resistant coating (*, **)
- Non-rotating handholds

Standard equipment for: *Model Alu / ** Model Country / ***Model Carbon

7. Maintenance tips for users**7.1. Cleaning and disinfection**

- The frame and the paneling of the wheelchair can be cleaned with a moist cloth. You can use a mild detergent for more stubborn dirt.
- The wheels can be cleaned with a wet synthetic fiber brush (do not use a wire brush!)
- Push handles, arm pads, and seat and back pads can be washed off. Use a mild detergent.
- Use a disinfectant listed by VAH for disinfection.

7.2. Rollator check by the user**7.2.1. Brake Inspection**

Make sure your brakes are operating properly before each use.

It should not be possible to push the rollator away while the brakes are activated.

The rollator should be easy to drive without grinding noises and with good direction stability when the brakes are released.

If you notice unusual behaviour when braking, inform your qualified mobility dealer immediately and do not use the rollator.

7.2.2. Checking the tyres

Check whether the tyres are intact every time before driving. If the tyre is damaged, it must be repaired by an authorised dealer.

8. Maintenance tips for retailers

If faults or defects at the rollator are found, it must be returned to the dealer for repair immediately.

The dealer can receive the required information and documentation for repair and service on request.

B+B customer service is available at +49 (0)700/60007070.

The rollator must be inspected in accordance to the service plan and must be in good condition.

9. Storage/shipping

If the rollator is to be stored or shipped, it is advisable to retain the original packaging.

Your rollator will have an optimum protection against environmental influences during storage and transport.

10. Transferring the rollator

The Alevo rollator is suitable for re-use. For further use, the rollator must first be thoroughly cleaned and disinfected in accordance with the attached care and maintenance instructions. Subsequently the product has to be inspected to establish its general condition and any wear or damage by an authorised dealer and repaired if necessary.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

11. Disposal

If the rollator is no longer used and you wish to dispose it, please contact your dealer.

If you wish to dispose the rollator yourself, please contact a local recycling company for the disposal regulations of your local jurisdiction.

12. Warranty

The warranty covers all defects of the rollator that are demonstrably due to material or manufacturing defects. Any claim must be accompanied by the completely filled-out warranty certificate.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of B+B will generally lead to forfeiture of warranty and product liability claims.

Our products come with a statutory guarantee of 2 years. For replacement devices and repairs, a warranty period of 1 year applies.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

Warranty-Certificate

Product: Alevo

Model number:

Serial number:*

* (to be filled in by authorized dealer)

Dealer:

Date and stamp

(Please note our general business terms and conditions)

*The serial number is located on the side frame.

Note!

Despite careful research and editing, incorrect information may slip into our operating manuals. Subject to technical changes. Figures may contain accessories. The most up-to-date version of the operating manual is available for download from our website:

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Bischoff & Bischoff GmbH
B+B France S.A.R.L.