Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

Senio

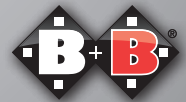
DE

EN

ES

FR

Enjoy mobility.

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Index

1.	Introduction	26	4.	Assembly/adjustments	34
1.1.	Signs and symbols	26	4.1.	Fitting push handles	34
2.	Important safety Instructions	27	4.2.	Unfolding	34
2.1.	General safety instructions	27	4.3.	Folding	34
2.2.	Indications	28	4.4.	Attach basket and tray	35
2.3.	Contraindications	28	4.5.	Adjusting the push handles	35
2.4.	Purpose	28	4.6.	Adjusting / activating the brakes	36
2.5.	Declaration of conformity	29	4.7.	Wheels and tyres	36
2.6.	Responsibility	29	5.	Use	37
2.7.	Service life	29	6.	Technical data	38
3.	Description of product and contents	30	7.	Care instructions for the user	40
3.1.	Inspection on delivery	30	7.1.	Cleaning	40
3.2.	Contents	31	7.2.	Disinfection	40
3.3.	Type plate and serial number	32	7.3.	Rollator check by the user	40
3.4.	Summary	33	7.3.1.	Brake Inspection	40
			7.3.2.	Checking the tyres	41
			8.	Maintenance tips for retailers	41
			9.	Storage/shipping	41
			10.	Transferring the rollator	42
			11.	Disposal	42
			12.	Warranty	43

1. Introduction

Dear user,

By purchasing a B+B rollator, Rollators Senio you are now the owner of a top-quality German product that is setting new standards in terms of its variability. This instruction manual is intended to help you become more familiar with the operation and use of the rollator.

Please read and follow the instruction manual carefully before using your new rollator for the first time. The instruction manual is an integral and necessary component of the product. Keep the instruction manual where you can refer to it, and if you transfer the rollator to another user, please provide the manual to the new owner.

For visually impaired users, this document is available as a PDF file at our website www.bischoff-bischoff.com.

Any repairs and adjustments require special technical training and may therefore only be carried out by authorized B+B dealers.

1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

2. Important safety Instructions

2.1. General safety instructions

- Max. load of Senio: 136 kg
- Pinch risk! Be careful when folding up and unfolding the rollator. Make sure that clothing and body parts are not pinched between the folding parts of the rollator.
- Observe the minimum insertion depth of the handholds.
- Each brake affects only one wheel.
- The basket can carry a weight of max. 5 kg.
- Do not use aggressive or caustic detergents to clean the rollator!
- Check that the rollator is fully intact each time before using it. Check that the brake is working.
- Have your mobility dealer check the brakes every six months!
- The surfaces of the rollator can heat up when exposed to direct sunlight. There is a risk of injury.
- Reporting incidents. The operator or user must immediately report
 1. any malfunction
 2. any change to the features or performance as well as
 3. any inaccuracies in the marking or operating instructions of a medical device, which has or could have led to death or to a serious deterioration of the health of a patient, an employee or a third party, to the Federal Institute for Drugs and Medical Devices (BfArM). This institute will immediately forward the notification to the operator's competent authority and also inform the manufacturer and the manufacturer's competent authority.

2.2. Indications

The rollator is designed for persons who have limited mobility but nevertheless have sufficient walking ability and strength to also move outside the home with the help.

2.3. Contraindications

The rollator should not be used with the following conditions:

- Disorders of perception
- Limb loss on the arms
- Severe equilibrium disorders
- Severely impaired vision
- Inability to sit

2.4. Purpose

The rollator supports mobility-impaired users with walking, e.g. at home, when shopping or walking outdoors. The seat allows the user to rest during longer periods of use.

Only use the rollator on a level and firm surface. The product may not be used as an aid for climbing or for transporting heavy loads.

The rollator is intended to increase the mobility of persons with impaired mobility or an equilibrium disorder.

The rollator can provide security when walking and additional support in daily life. The ability to transport smaller loads, up to 5 kg, and to sit down when tired, makes it easier to perform everyday activities.

Please note: We can only offer a warranty if the product is used in accordance with the specified conditions and for the intended purposes.

2.5. Declaration of conformity

B+B, as manufacturer and solely responsible party, hereby declares that the rollator Senio complies with the requirements of the EC Directive 93/42.

2.6. Responsibility

We are only able to provide a warranty if the product is used in accordance with the specified conditions and for the intended purposes. If modifications, repairs or maintenance works are made, please make sure only authorised persons perform these works in accordance to the instructions of B+B.

2.7. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time the rollator can be used continuously, provided it is in a safe condition.

3. Description of product and contents

3.1. Inspection on delivery

All B+B rollators are subject to a proper final inspection in our house and receive the CE mark.

The rollator is supplied in a special box. After unpacking please save the box if possible. You may need it later for storage or to return the product.

Assembly, adjustment to your body size and first instructions are taken care of by your qualified mobility dealer.

If delivered by rail or a forwarding agent, the product is to be checked with regard to transportation damage at once in the presence of the shipper.

Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service (Tel.: +49 700 6000 7070).

3.2. Contents

When you receive the product, please check immediately that the contents are complete. The contents comprise:

- Outer packaging
- Rollator
- Two sliding tubes
- Two star grip nuts for fixing and adjusting the sliding tubes
- Instruction manual
- Basket
- Tray

3.3. Type plate and serial number

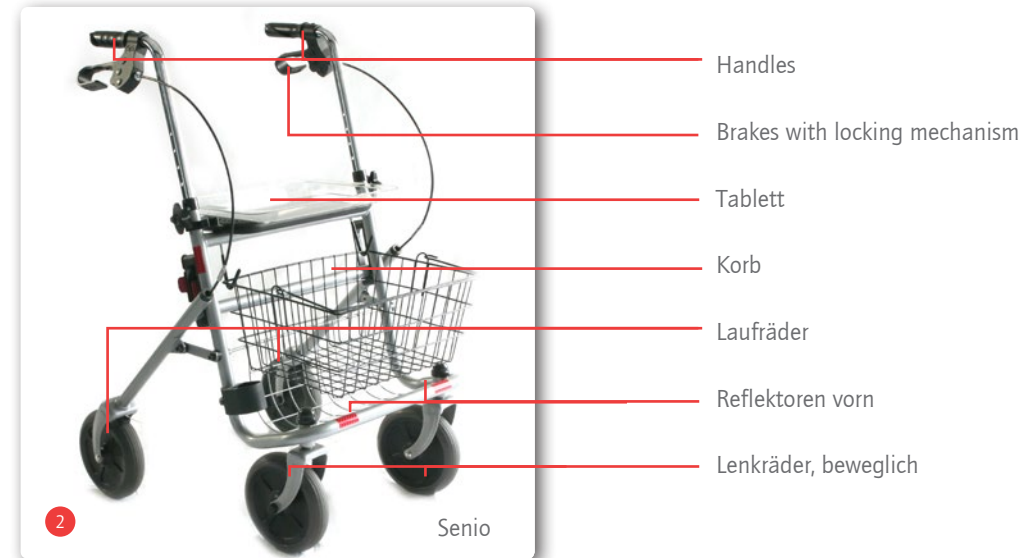
The type plate (Fig. 1) and serial number are located on the bottom of the frame.

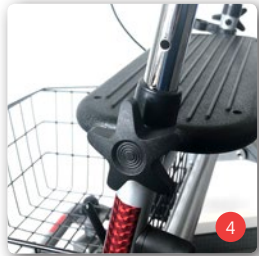
- A Manufacturer's logo
- B Manufacturer's information
- C Caution! - Follow instructions for use
- D CE mark
- E Important! - Follow instructions for use
- F Model designation
- G Application class
- H Model number
- I Date of manufacture
- J max. height of the push handles
- K Maximum load capacity
- L Required dimensions
- M UDI number
- N Serial number

3.4. Summary

Product overview Rollator Senio (Fig 2) and (Fig. 3):

4. Assembly/adjustments

4.1. Fitting push handles

- Insert the push handles in the handle tubes and slide the screw through the holes from inside to outside.
- First, slide the curved plastic adapter onto the screw so that it conforms to the tube. Then push the washer onto the screw (Fig.10).
- Place the star nut on the screw and tighten (Fig.11).
- To set the correct height, read section 4.5

4.2. Unfolding

- Position the rollator in the travel direction.
- Release the locking brakes (see sec. 4.5).
- Tip the B+B rollator towards yourself until the load is taken off the steered wheels and press the handles downwards at the same time with a gentle jolt.
- To lock the folding mechanism, now fold the seat plate tight to your seat (Fig. 4). This locks the rollator.

4.3. Folding

- Position the rollator in the travel direction.
- Activate the locking brakes.
- To unlock, pull the unlocking mechanism to the left on the right-hand side (Fig. 7).
- Hold the rollator firmly with both hands and tilt it by pulling it towards you.

Note that the rollator may suddenly fold up when the lever is activated.

- For unlocking, take one hand off the handle and use it to activate the lever of the auto-lock system (Fig. 7). Ensure that neither body parts nor pieces of clothing become pinched during the folding process.
- To fold the rollator up completely, pull the handles towards you.


4.4. Attach basket and tray


- The rollator is unfolded. Activate the locking brakes.
- Position yourself against the travel direction in front of the rollator.
- Simply hang the basket onto the hooks attached to the front frame tubes (Fig. 8)
- To remove the basket, simply hold the carrying handle in your hand and move it upwards to remove
- Place the tray onto the seat from above in such a way that the four plastic pins of the tray fit into the holes of the seat (Fig. 9).

4.5. Adjusting the push handles

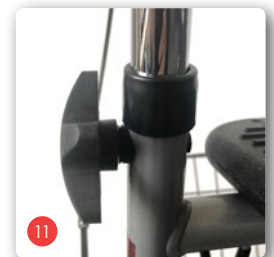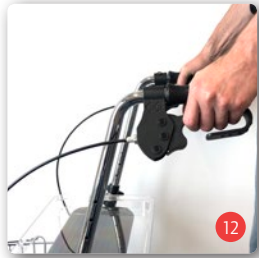- The B+B rollator is unfolded. Activate the locking brakes.
- Position yourself behind the B+B rollator and let your arms hang down in a relaxed manner.
- Now adjust the push handles so that they are at the level of your wrists.
- Push the screw from inside to outside through the hole closest to this height. Place the curved plastic adapter and then the washer on the screw (Fig.10).
- Place the star nut on the thread and tighten (Fig. 11).
- Repeat this procedure on the other side.
- Check to make sure that both the push handles are at the same height.

4.6. Adjusting / activating the brakes

Service brake

To activate the service brake, pull both handbrake levers upward on the push handles upward (Fig. 12). Service and locking brake act on the fixed wheels.

Locking brake

- Press down on both brake levers to activate the locking device (Fig. 13).
- To release the locking brake pull both brake levers back into middle position (Fig 14).

Adjusting the brakes

We recommend carrying out the brake adjustments together with your specialist dealer. If you have the necessary technical knowledge, you can also make the adjustments yourself.

The brake is set properly, if the brake pins do not touch the wheel and the locking brake can be activated easily.

- Loosen the locknut (Fig. 15, arrow) so the adjustment nut can turn.
- Now turn the wide adjustment nut further inwards or outwards to increase or decrease the braking effect.
- Retighten the locknut.
- Repeat the procedure on the other brake if necessary.

4.7. Wheels and tyres

Your rollator is equipped with flat-proof PU wheels 200 x 50 mm.

5. Use

Walking

Make your first attempts to walk with the rollator on a flat, firm surface. Familiarise yourself with the function of the brakes. Let someone accompany you during this initial period.

For better visibility, there are reflectors on the sides, front and back (Figs. 16-17).

For an anatomically correct gait, the push handles must be set at proper height (see sec. 4.6.).

- Place yourself in the rollator so that your ankles are at the height of the rear wheels.
- Stand with good posture.
- Grasp the push handles with both hands and your arms bent at a slight angle.

When walking with the rollator try to stay as close as possible with the body to the push handles. This increases the stability and safety against tipping over; in addition, avoid a crooked back. Try to generally walk in the rollator and not behind it. Take small steps and do not push the rollator too far ahead.

You can determine the change of direction and speed by pushing the rollator. Always use the brakes to slow down your movement. The rollator could otherwise turn with the unbraked wheel and the braked wheel, and could fall over.

Go around obstacles whenever possible. Try to use ramps or take lifts. If this is not possible, the rollator must be carried over the obstacle. Find a second person to help you do this, if possible.

To negotiate small obstacles such as curbs, use the tip assist (Fig. 18) to tip the rollator (Fig.19).

The seat

The rollator is also designed to be used as a seat by the user for short breaks. It is therefore equipped with a seating area. You must absolutely activate the locking brakes before sitting down. The rollator could roll away and you could fall.

Lifting points

You can lift the rollator on the grips or frame sections, in order to carry it over obstacles or steps.

6. Technical data**Dimensions and weights**

Type:	Rollator
Colour:	Silver
Model:	4004
Height (push handle height):	max. 980 mm
Length:	680 mm
Folded (H x L x W):	700 x 590 x 245 mm
Width:	590 mm
Recommended height:	59 cm - 190 cm.
Width between handles:	430 mm
Width of handles:	33 mm
Angle of push handles max.:	0°
Seat height:	610 mm
Seat width:	380 mm
Effective seat width:	420 mm
Turning radius:	815 mm
Weight:	9,8 kg, including basket and tray
Net weight without accessories:	9,0 kg
Max. load:	136 kg
Max. load of bag:	5 kg
Pipe diameter frame:	26 mm

Tyres

Rear wheels:	192 x 48 mm
Steered wheels:	192 x 48 mm

Material

Frame:	steel, corrosion-resistant coating
Handles:	PP / TPE SEPS
Seat:	PU
Flammability:	flame-retardant

Operating conditions

Temperature:	0 to 50°C
Air humidity:	45 to 80%

Storage conditions

Temperature:	-20 to + 60°C
Air humidity:	45 to 60%

Standard equipment for the Senio rollator

Includes basket and tray
Ergonomically-shaped handles
Combined service and locking brake
Tipping aid on the right wheel
Two reflectors on each side, as well as front and back on the frame
Stable steel design, corrosion-resistant coating.

7. Care instructions for the user

7.1. Cleaning

- The frame components of the rollator can be wiped with a damp cloth. You can use a standard mild, soap-based cleaning agent if they are very dirty.
- The wheels can be cleaned with a damp brush with plastic bristles (do not use a wire brush).
- Push handles, armrests and seat cushion can be wiped off. Again, use a mild household cleaning solution.
- Then dry your rollator with a soft cloth (e.g. tea towel).
- Do not use a high-pressure cleaner

7.2. Disinfection

- The rollator must be thoroughly cleaned prior to disinfection.
- For disinfection, use a standard disinfectant. Please follow the instructions of the manufacturer regarding handling such as exposure time, etc.)

7.3. Rollator check by the user

7.3.1. Brake Inspection

Make sure your brakes are operating properly before each use.

It should not be possible to push the rollator away while the brakes are activated.

The rollator should be easy to drive without grinding noises and with good direction stability when the brakes are released.

If you notice unusual behaviour when braking, inform your qualified mobility dealer immediately and do not use the rollator.

7.3.2. Checking the tyres

Check whether the tyres are intact every time before driving. If the tyre is damaged, it must be repaired by an authorised dealer.

8. Maintenance tips for retailers

We recommend that the brakes of the rollator be inspected by the mobility dealer every six months.

If faults or defects at the rollator are found, it must be returned to the dealer for repair immediately.

The dealer can receive the required information and documentation for repair and service on request. A training program is also available for dealers.

B+B customer service is available at +49 (0)700/60007070.

9. Storage/shipping

If the rollator is to be stored or shipped, it is advisable to retain the original packaging.

Your rollator will have an optimum protection against environmental influences during storage and transport.

10. Transferring the rollator

The rollator Senio is suitable for re-use. For further use, the rollator must first be thoroughly cleaned and disinfected in accordance with the attached care and maintenance instructions. Subsequently the product has to be inspected to establish its general condition and any wear or damage by an authorised dealer and repaired if necessary.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

The rollator must be inspected in accordance to the service plan and must be in good condition.

11. Disposal

If the rollator is no longer used and you wish to dispose it, please contact your dealer.

If you wish to dispose the rollator yourself, please contact a local recycling company for the disposal regulations of your local jurisdiction.

12. Warranty

The warranty covers all defects of the rollator that are demonstrably due to material or manufacturing defects.

Any claim must be accompanied by the completely filled-out warranty certificate.

Our products come with a statutory guarantee of 2 years. For replacement devices and repairs, a warranty period of 1 year applies.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of B+B will generally lead to forfeiture of warranty and product liability claims.

Warranty-Certificate

Product: Senio

Serial number: *
*(to be filled in by authorized dealer)

Dealer:

Date and stamp
(Please note our general business terms and conditions)

*The serial number is located on the side frame.

Note:
No liability is accepted for printing errors, product and colour modifications, or changes to technical specifications. The colours of the product shown may vary slightly in reality. Figures may contain accessories.

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Bischoff & Bischoff GmbH
B+B France S.A.R.L.