Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

BX 11
BX 11 S

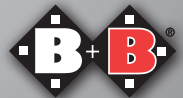
DE

EN

ES

FR

Enjoy mobility.

Inhalt

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Contents

1.	Introduction	48
1.1.	Labels and symbols	48
2.	Important safety information	50
2.1.	General safety notes	50
2.2.	Risk of tipping over	52
2.3.	Safety in vehicles	52
2.4.	Use on roads	52
2.5.	Intended use	52
2.6.	Indications	52
2.7.	Contraindications	53
2.8.	Declaration of conformity	53
2.9.	Liability	53
2.10.	Service life	53
3.	Product and scope of delivery	54
3.1.	Checking the delivered goods	54
3.2.	Scope of delivery	54
3.3.	Product's identification label and serial number	55
3.4.	Overview	56
4.	Assembly/Adjustments	58
4.1.	Folding und Unfolding	58
4.2.	Legrests	58
4.2.1.	Footplate flip-up	58
4.2.2.	Swing-away legrests	60
4.2.3.	Horizontally-adjustable legrests (optional)	61
4.3.	Amputee legrests (optional)	62
4.3.1.	Active mudguard	62
4.3.2.	Height-adjustable sideguards with armrests (optional)	62
4.3.3.	Detachable sideguards with height-adjustment (optional)	63
4.4.	Backrest angle	63
4.4.1.	Backrest (standard)	63
4.4.2.	Angle-adjustable backrest (optional)	64
4.5.	Push handle height	64
4.5.1.	Push handles (standard)	64
4.5.2.	Push handles (optional)	64
4.6.	Adjustable seat and backrest	64

4.6.1.	Seat slack	64
4.6.2.	Back slack	65
4.7.	Brakes	65
4.7.1.	Operating the parking brake	66
4.7.2.	Operating the brake lever extension (optional)	66
4.7.3.	Drum brake (optional)	67
4.8.	Wheels and tyres	68
4.8.2.	Drive wheels	68
4.8.3.	Castors	68
4.9.	Anti-tip wheels and tipping aids (optional)	69
4.9.1.	Tipping aids (optional)	69
4.9.2.	Anti-tip wheels (optional)	69
4.10.	Camber	70
4.11.	Spokeguard (optional)	70
4.12.	Fitting the bag (optional)	70
5.	Use	71
5.1.	Transferring from the side	71
5.2.	Transferring from the front	71
5.3.	Going up or down stairs or a high step	72
5.4.	Driving the wheelchair	73
6.	Technical data	74
7.	Transport	76
7.1.	Transporting the wheelchair	76
7.2.	Transportation in vehicles	76
8.	Maintenance instructions for the user	76
8.1.	Cleaning and disinfection	76
8.2.	Checks before driving	77
8.2.1.	Checking the brakes	77
8.2.2.	Checking the tyres	77

9.	Adjustment work for the specialised retailer	77
9.1.	Centre of gravity, seat angle and seat height	77
9.1.1.	Adjusting the drive wheel	77
9.1.2.	Height of the front wheel	78
9.1.3.	Angle of the steering axle	78
9.1.4.	Wheelbase	79
9.2.	Adjusting the parking brakes	79
9.3.	Adjusting the drum brakes	80
9.4.	Adjusting the anti-tip wheels	80
10.	Repairs	80
11.	Maintenance information for specialised retailers	81
12.	Passing the wheelchair on to another user	81
13.	Storage/shipping	82
14.	Disposal	82
15.	Warranty	82

1. Introduction

Dear user,

You have chosen a high-quality wheelchair from B+B. Thank you for trusting us.

This user manual contains all the information necessary to operate both BX11 with fixed frame, one piece footplate and with swing-away legrests

The wheelchair has been designed to offer many advantages and meet your needs. Through the wide range of adjustment options and the very best adaptability with regards to seat width, height, depth, back-rest height etc., the BX 11 / BX 11 S can be perfectly adapted to your individual body measurements.

The wheelchair was designed for those active users who rely on wheelchair for both in- and outdoor activities.

Before using your new wheelchair for the first time, please read and follow the information in this User manual carefully. This is an integral and necessary part of the wheelchair. Keep the User manual within easy reach and pass it on when transferring the wheelchair.

For users who are visually impaired, this document is available as a pdf file on our website www.bischoff-bischoff.com.

Note!

Despite thorough research and editing, incorrect information may occur in our User manual. We reserve the right to make technical modifications. Images may show accessories. You can find the current version of the User manuals in the download area of our website:

www.bischoff-bischoff.com

Repair and adjustment work require special technical training and must therefore only be carried out by dealers who have been authorised by Bischoff & Bischoff GmbH.

1.1. Labels and symbols

Important! Indicates particularly useful information in the relevant context.

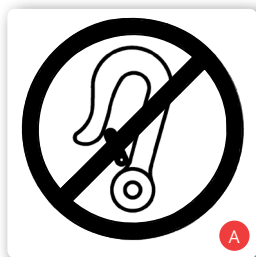
Attention! Means particular safety-related Notes. Observe the User manual!

2. Important safety information

2.1. General safety notes

- In order to prevent falls and dangerous situations, practise with your new wheelchair on level, clear ground first. An attendant is recommended.
- Before using the wheelchair check that all attached parts are properly fixed.
- Only lift the wheelchair using firmly installed parts (see Chapter 5.3).
- Make sure that the quick-release axles are firmly positioned in the axle support, otherwise there is a risk of injury.
- The risk of tipping over to any side increases with changes in balance through body movements or loading the wheelchair.
- The wheelchair must not be used to transport several individuals or loads. Please respect the intended use (see Chapter 2.5)!
- When carrying out setting or adjustment work on the wheelchair there is a risk that body parts could get pinched!
- Keep items of clothing and body parts away from gaps between moving parts!
- When overcoming obstacles (steps, etc.) call in an attendant. Use tipping aids (see Chapter 4.9.1).
- Negotiate steps and large obstacles with the help of at least two attendants (see Chapter 5.3). Use equipment such as ramps, lifts or stair-climbing devices, if these are available.
- When driving on downhill/uphill slopes, please be aware that even at a few degrees, there is a risk of tipping over. We recommend the use of anti-tip wheels (see Chapter 4.9.2).
- Avoid collisions of any type!
- Drive only on level, firm ground
- When transferring to or from the wheelchair, apply both parking brakes (see Chapter 4.7).
- When on the road, observe the road traffic regulations (StVZO).

- Each time, before setting off, check the passive lighting (reflectors) of your wheelchair to make sure they are not damaged and are visible for other road users.
- The parking brakes are to be used only while parking.
- The seating materials are flame resistant in compliance with EN 1021-1, EN 1021-2. However, please keep cigarettes and other possible ignition sources away from your product.
- Protect the wheelchair from sunlight to avoid burns caused by heated components
- Protect the wheelchair from the cold
- When transferring between the wheelchair and the bed (see Chapter 5.1, 5.2) always
 - apply both parking brakes
 - Swing the sideguard on the transfer side to the back if this is possible.
 - flip the footplates up or swing them away.
- Notification of incidents. The provider or user must report
 1. Any malfunction
 2. Any change in the characteristics or performance as well as
 3. Any inadequacy in the labelling orinstructions for use of a medical product, which has led or could lead to death or a severe deterioration in the health condition of a patient, an employee or a third party, must be reported immediately to your appropriate state authority responsible for medical products. They will pass the report on to the authority responsible for the provider immediately and will also inform the manufacturer and the authority responsible for the manufacturer.

2.2.Risk of tipping over

If you have anti-tip wheels fitted on both sides (see Chapter 4.9.2) these will largely prevent the wheelchair from tipping over backwards. However please note that driving with anti-tip wheels is only possible in a limited way on ground with different levels, lift platforms and ramps. Make sure there is sufficient gap at the top, bottom and sides.

2.3.Safety in vehicles

The BX 11 / BX 11 S is not designed to be used as a vehicle seat. Under no circumstances must it be used as such. It does not fulfil the requirements in compliance with ISO 7176-19. Violation can cause the most serious injuries resulting in death in the case of an accident (Fig. A).

2.4.Use on roads

The wheelchair was designed for both in- and outdoor activities. Please note that when you drive on public roads. Do not endanger other users through reckless driving, this applies particularly on pavements.

2.5.Intended use

The wheelchair is only designed to increase mobility for those who have difficulty walking with the indications specified (see Chapter 2.6). The maximum payload is 135 kg.

Please note:

Warranty will be granted only if the product is being used according to the specified conditions and for the intended purposes.

2.6.Indications

Provision of a wheelchair is intended for individuals who are unable to walk or who have severely impaired mobility, e.g. through:

- Paralysis
- Loss of limbs
- Limb defects/deformity
- Joint contractions (not on both arms)
- Joint damage (not on both arms)
- other illnesses

2.7. Contraindications

Provision of wheelchairs is not suitable for individuals who have:

- No capability to sit
- Joint damage/contractions on both arms
- Perceptual disorders
- severe equilibrium disorders
- reduced and insufficient vision
- severe limitations of cognitive abilities

2.8. Declaration of conformity

We, Bischoff & Bischoff GmbH, declare under sole responsibility, that the BX 11 / BX 11 S wheelchair meets all requirements of MDR 2017/745, which are applicable.

2.9. Liability

Warranty will be granted only if

- the product is being used according to the specified conditions and for the intended purposes.
- Modifications, enhancements, repairs and maintenance work is carried out only by those individuals who have been authorised by us to do so, and the modifications or enhancements are approved by B&B and
- the wheelchair is used in accordance with the instructions for use.

2.10. Service life

The expected product service life is up to 5 years. Prerequisites for this are proper use as well as compliance with the safety, care and maintenance information in accordance with this user manual.

3. Product and scope of delivery

3.1. Checking the delivered goods

All products from Bischoff und Bischoff GmbH undergo an appropriate final inspection at our factory and the CE mark is affixed (see Chapter 3.3).

The wheelchair will be delivered in a special cardboard box. After unpacking, retain the cardboard box if possible. It can be used for possible later storage or return of the product.

Delivery and induction is usually carried out by a qualified medical equipment retailer.

When sending by train or shipping company, the goods must be checked immediately for transport damage in the presence of the forwarding agent.

Check that the delivery is complete and undamaged (see Chapter 3.2). In the case of anomalies or damage, contact our Customer service, Tel.: +49 (0)700/6000 7070.

3.2.Scope of delivery

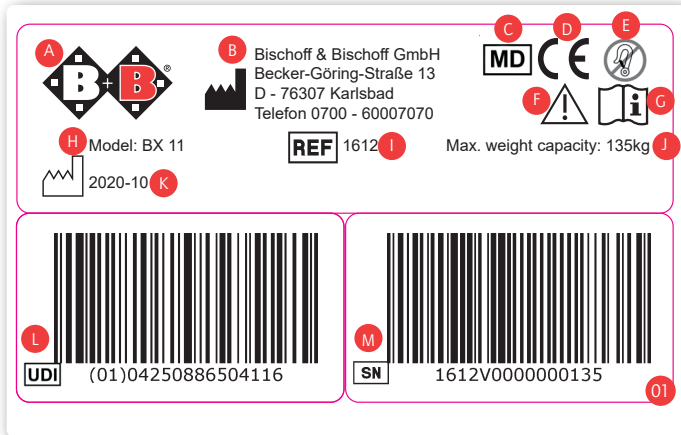
In order to avoid transport damage, detachable attachments are delivered separately with the product.

On receiving the goods, please immediately check that the delivery is complete. The delivery consists of:

- Packing,
- Wheelchair (preassembled),
- Back cushion
- 2 legrests (for BX 11 S)
- this user manual
- any necessary accessories

3.3. Product's identification label and serial number

The product's identification label (Fig. 01) and serial number are located under the cross brace.

- A** Manufacturer's logo
- B** Manufacturer information
- C** MD Medical product
- D** CE mark
- E** Observe safety in vehicles
- F** Attention! - read the User manual
- G** Important! - read the User manual
- H** Model name
- I** Model number
- J** max. payload
- K** Date of manufacture
- L** UDI number
- M** Serial number

3.4.Overview

BX 11 S (Fig. 02)

- A** Push handles
- B** Backrest upholstery
- C** Castors
- D** Active mudguard
- E** Seat upholstery
- F** Drive wheel
- G** Quick-release axle
- H** Parking brake
- I** Handrims
- J** swing-away legrests
- K** one piece footplate

BX 11 (Fig. 03)

4. Assembly/Adjustments

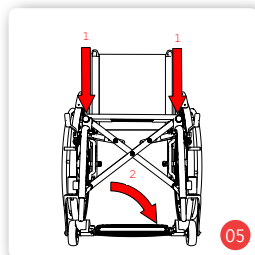
4.1. Folding und Unfolding

Your new B+B wheelchair will be delivered in a B+B original cardboard box, assembled and folded.

In order to avoid transport damage, detachable attachments are delivered separately with the product.



Unfolding

- Position the folding wheelchair next to you. (Fig. 04)
- Tip the wheelchair toward until the weight is off one of the drive wheels.
- Push the seat tubes apart and push them down with the flat of your hand, until they lock into place in the bracket on the frame (Fig. 05).
- Push the footplate down (Fig. 05).

Folding

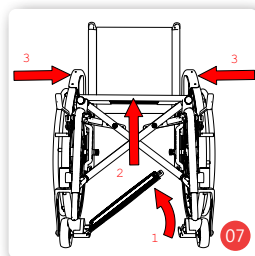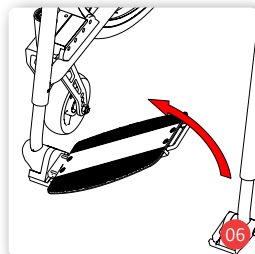- Position the folding wheelchair next to you.
- Fold the footplate up (Fig. 06, Fig. 12).
- Undo the hook and loop fastener on the back sling (under the seat).
- If using a seat cushion, remove it, push the seat sling upwards and then push the armrests together as far as this is possible (Fig. 07).

When folding and unfolding, watch your fingers as there is a risk of pinching

4.2. Legrests

4.2.1. Footplate flip-up

The BX 11 is fitted with a one piece, flip-up footplate (Fig. 08).

You can flip the footplate up to the side, adjust the angle and adjust it to the lower leg length of the user.

Flipping the footplate up and down

- Grip under the footplate and flip it up to the side (Fig. 09).
- Once you are sitting in the wheelchair, flip the footplate down again. Then press the footplate down until it you hear it click into place.

Adjusting the legrest length

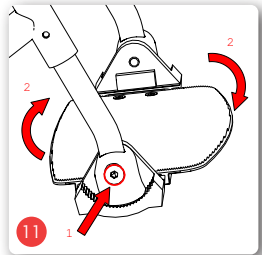
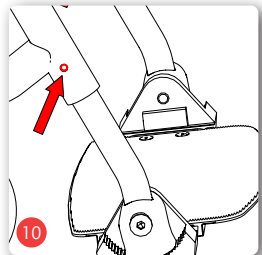
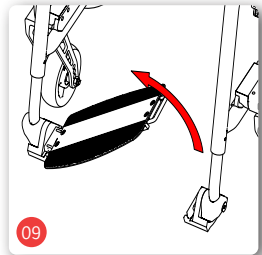
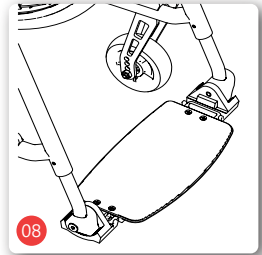
The footplate can be adjusted to the lower leg length in 8 positions at 15 mm steps.

- To do this remove both socket screws on the legrest tube (inner and outer) and adjust to the required length (Fig. 10).
- Re-tighten the screws.

Adjust the angle of the footplate

The angle of the footplate can be adjusted (Fig. 11):


- Unscrew the socket screw.
- Twist the sprocket to adjust the angle of the footplate to the required position
- Re-tighten the screw.

Before driving, please make sure that the legrests are firmly in place!

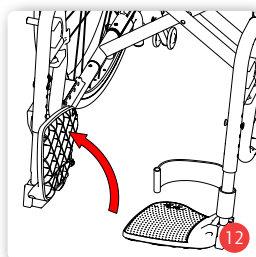Carry out the same adjustments on both sides.


4.2.2.Swing-away legrests

The BX 11 S is fitted with removable legrests (Fig. 12). You can flip the footplates up separately to the side, adjust the angle and adjust to the lower leg length of the user. You can remove the legrests, swing them away and refit them without needing to use tools.

Attach legrests


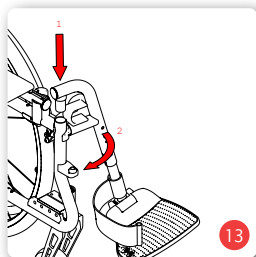
- Hold the top of the legrest so that it is pointing outwards.
- Guide the legrest into the hole in the frame using the guide profile (Fig. 13).
- Swing the legrest to the front until you hear it clearly click into place (Fig. 13)
- Flip the footplate down if necessary.

Swing the legrests away and remove them

In order to make it easier to transfer to and from your folding wheelchair, you can swing the legrests inwards or outwards, without having to remove them.

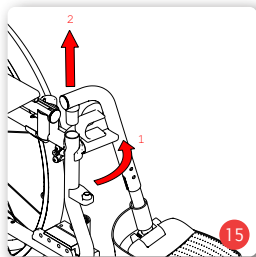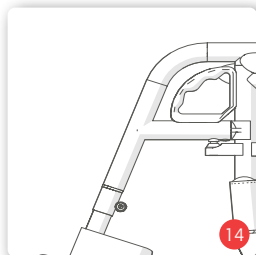- Flip the footplate up if necessary.
- Pull the handle upwards (Fig. 14) and swing the legrest outwards (Fig. 15).
- In the outward position, the legrest can be pulled out by pulling it upwards (Fig. 15).

Before setting off, fit the legrests as described above.

Never stand on the footplate!

Adjusting the legrest length

The footplates can be adjusted to the lower leg length in 5 positions

Before driving, please make sure that the legrests are firmly in place!

at 20 mm steps.

- To do this remove both socket screws on the legrest tube (inner and outer) and adjust to the required length (Fig. 16).
- Re-tighten the screws.

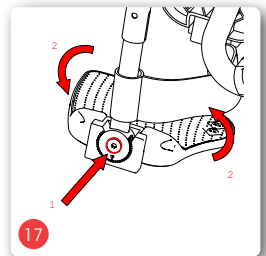
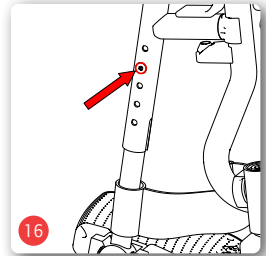
Angle-adjustable footplates

The angle of the footplates can be adjusted (Fig. 17).

- Unscrew the socket screw.
- Twist the sprocket to adjust the footplate to the required position.
- Re-tighten the screw.

4.2.3. Horizontally-adjustable leg-rests (optional)

Assemble, disassemble and swing them away as for the standard leg-rests.

Carry out the same adjustments on both sides.

To adjust, undo the clamp lever, then adjust the legrest to the required length and then tighten the clamp lever

The horizontal adjustment works as follows:

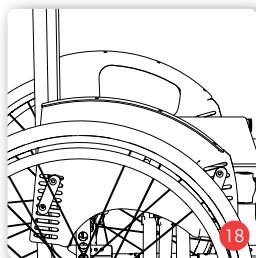
- Push the locking lever forwards.
- by lifting or lowering the footplate receiver tube you can get the legrest into the required position.
- Release locking lever; legrest locks into place automatically.

Never stand on the footplate and when swinging it away, watch your fingers as there is a risk of pinching

To change seat, the legrest must be swung away or removed. Never sit on the legrest. Risk of tipping over!

4.3.Amputee legrests (optional)

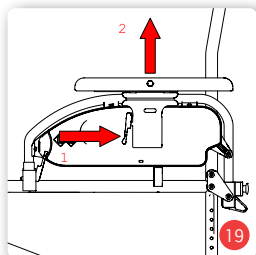
Assemble, disassemble and swing them away as for the standard legrests, adjustment and the horizontal adjustment works as for the horizontally-adjustable legrests.

4.3.1.Active mudguard

As standard the wheelchair is fitted with an aluminium mudguard (Fig. 18). This is fixed to the frame at the front and the back. The height of the mudguard must be adjusted if the seat position or tyre size is changed.


The dealer can adjust the mudguards by using the assembly instructions.

4.3.2.Height-adjustable sideguards with armrests (optional)

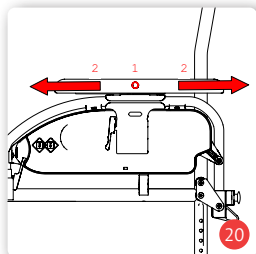The height-adjustable sideguards are fitted with height and depth-adjustable armrests. You can swing the sideguard away and remove it completely.

Armrest height

- Operate the lever with one hand and push the armrest to the required height with the other (Fig. 19).
- Release the lever so that the armrest clicks into the next possible position.

Armrest depth

- Push the pushbutton on the outside of the armrest and push the armrest forwards or backwards (Fig. 20).
- Release the push button so that the armrest clicks into the next possible position.

Swinging the sideguards away

- Operate the lever and at the same time swing the sideguard away to the back (Fig. 21).

Removing the sideguards

- Swing the sideguard upwards (Fig. 21) by holding the armrest.
- With the other hand pull the catch on the rear fixing out. Now pull the sideguard upwards to remove it (Fig. 22)

To assemble first push the sideguard to the rear catch, until you hear the catch lock into place. Then flip the sideguard forwards, until it clicks into the front lock.

4.3.3. Detachable sideguards with height-adjustment (optional)

The detachable sideguards are fitted with height-adjustable armrests. You can remove the sideguard completely.

Armrest height

- Push the pushbutton on the lower end of the tube and adjust the armrest to the required height (Fig. 23).
- Release the push button so that the armrest clicks into the next possible position.

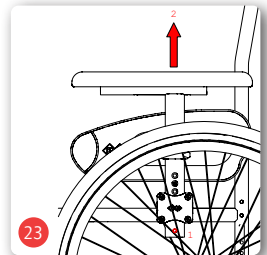
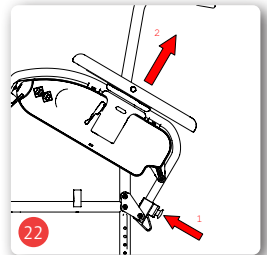
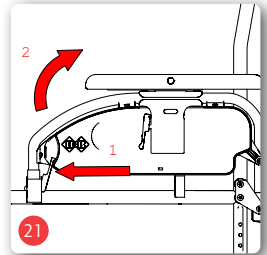
Removing the sideguards

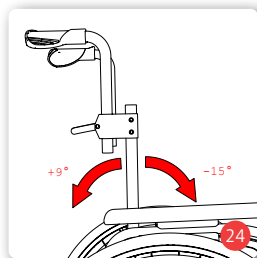
- Push the pushbutton on the lower end of the tube and pull the sideguard fully out of the lock (Fig. 23).

To assemble, push the sideguard into the lock and adjust the armrest to the required height. You must hear the sideguard click into place.

4.4. Backrest angle**4.4.1. Backrest (standard)**

The backrest of your wheelchair cannot be adjusted. The backrest angle is 90° to the seat surface.


4.4.2. Angle-adjustable backrest (optional)

The optional backrest of the wheelchair can be angle-adjusted (Fig. 24).

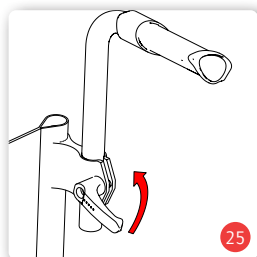The dealer can adjust the backrest by using the assembly instructions.

4.5. Push handle height

4.5.1. Push handles (standard)

The push handles can be fitted in a comfortable position for the attendant. Then adjust the backrest height and back slack.

The dealer can use the assembly instructions for further adjustments.

4.5.2. Push handles (optional)

The optional push handles can be adapted to suit the individual body size of the attendant, regardless of other adjustments.

- Release the lock (Fig. 25).
- Adjust to a comfortable height for the attendant.
- Re-tighten the lock.

4.6. Adjustable seat and backrest

4.6.1. Seat slack

The seat surface of your wheelchair can be re-tensioned. Always make sure it is sufficiently taut. However make sure there is sufficient slack in the pelvic area.

- If fitted, first remove your seat cushion.
- Undo the hook and loop fastener under the seat surface

- Pull the seat sling as tight as you need and fix in place using the hook and loop fastener (Fig. 26).
- Put your seat cushion back on the seat surface.

4.6.2. Back slack

A well-adjusted backrest will mean that you can sit for long periods of time more comfortably and reduces the risk of pressure sores. When doing this make sure there is sufficient slack in the pelvic area!

- First remove the cushion support of the backrest sling completely (Fig. 27).
- Undo the hook and loop fastener on the straps (Fig. 28).
- Adjust the straps in sequence, starting from the bottom
- Put the cushion support back over the tension straps.

4.7. Brakes

As standard, the wheelchair is fitted with parking brakes to be operated by the wheelchair user. Optionally, you can expand the system add a drum brake, which is operated by an attendant.

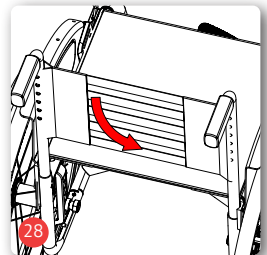
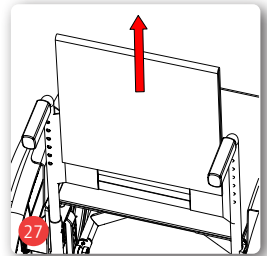
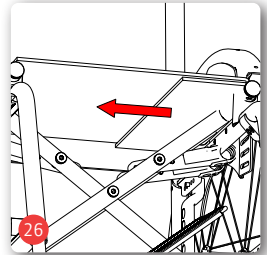
The dealer can adjust the drum brake by using the assembly instructions.

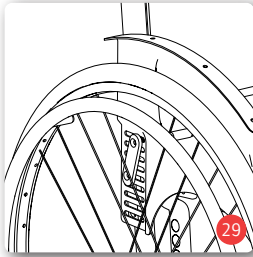
To adjust the brakes, contact your dealer. Please bear in mind that if the brakes are set incorrectly this can have life-threatening consequences.


The parking brakes must not be used as operational brakes, as there is a risk of death due to the risk of falling!

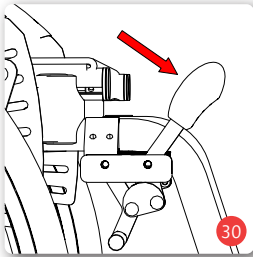Please note that there is a risk of burns if your hands heat up when braking using the handrims!

4.7.1. Operating the parking brake

The parking brake must only be used for parking. To slow down while driving, you must only use the drive wheel handrims (Fig. 29) or the optional drum brakes for attendant.

- When braking using the handrims, hold the highest point. First let the handrims slide through your hands
- Slowly increase the braking force by exerting holding force on the handrims, until you come to an abrupt stop.
- As soon as the wheelchair has stopped, push both brake levers forward (Fig. 30). The wheelchair is securely braked, if it can no longer be pushed and both brake levers are pushed forward to the stop position.
- Pull the lever towards you to release the brake.

4.7.2. Operating the brake lever extension (optional)

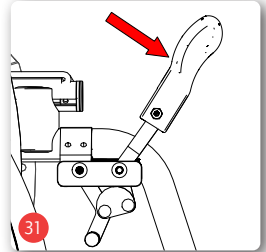
As an option, your wheelchair can be fitted with a brake lever extension. The extended lever reduces the effort required and is therefore particularly recommended for individuals with reduced arm and hand strength.

- Bring the wheelchair to a standstill (see Chapter 4.7).
- Push both brake levers forward. The wheelchair is securely braked, if it can no longer be pushed and both brake levers are pushed forward to the stop position (Fig. 31).
- Pull the lever towards you to release the brake.


Operational position Extend the brake lever to its longest position and fold it down before you set off (Fig. 32, Fig. 33).


Fold the brake lever to the rear when transferring to the side (Fig. 33).

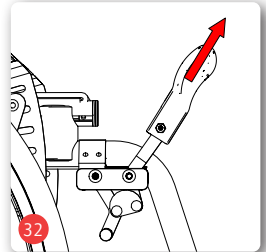
The dealer can adjust the brake and assemble the brake lever extension by using the assembly instructions.

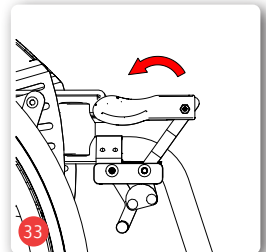The parking brake - with or without brake lever extension - only works on one drive wheel and must therefore always be used on both sides!

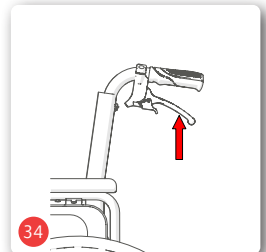
Never use the brake lever as a support - e.g. when transferring or standing up -, otherwise there is a risk of tipping over and falling! The lever could break off!

4.7.3. Drum brake (optional)

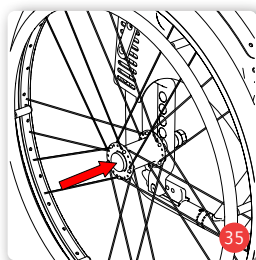As an option, the wheelchair can be fitted with a drum brake which can only be operated by an attendant using the brake lever on the push handles (Fig. 34).

4.8. Wheels and tyres

4.8.1. Quick-release axle

As standard the wheelchair has quick-release axles. These can therefore be easily removed and refitted to the wheels.

- Push the lock of the quick-release axle using your thumbs and pull the wheel off (Fig. 35).
- When refitting, simply put the wheel onto the receiver. By doing this, push the locking knob in.

The click you will hear from the quick-release axle will indicate that the wheel is securely mounted. After fitting, make sure that the wheels are tight!

4.8.2. Drive wheels

Check the drive wheels regularly to make sure there is sufficient air pressure and tread.

Pump the wheel to the pressure according to the value on the tyre's wall.

4.8.3. Castors

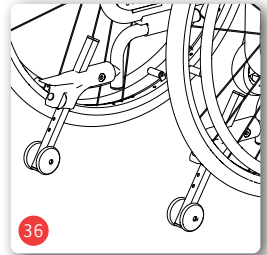
If the drive wheel or castor is worn or defective, ask your dealer to service them!

4.9. Anti-tip wheels and tipping aids (optional)

The combination of anti-tip wheels and tipping aids (Fig. 36) can be used for greater stability on slopes. In addition, an attendant can help to overcome obstacles, such as, for example, kerbs.


The dealer can make the adjustment by using the Assembly instructions.

4.9.1. Tipping aids (optional)

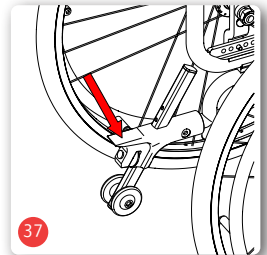- Tipping aids can be fitted on one or both sides.
- Hold the wheelchair firmly with both hands on the push handles.
- Put one foot on the tipping aids (Fig. 37) and at the same time push the wheelchair down using the push handles.

4.9.2. Anti-tip wheels (optional)

The anti-tip wheels prevent the wheelchair from tipping over backwards.

Anti-tip wheels are particularly recommended for inexperienced wheelchair users or amputees.

For safety reasons, always use the anti-tip wheels as a pair!

For adjustment, contact your dealer

4.10. Camber

If your wheelchair has drum brakes, there is an option of adjusting the camber.

For adjustment, contact your dealer

4.11. Spokeguard (optional)

The spokeguard is being fixed to the spokes with cable ties.

4.12. Fitting the bag (optional)

If you want to fit a bag to your wheelchair, hang the straps of your bag over the back tubes of your wheelchair.

Please note that the reflectors must not be covered.

Attention - increasing the load in the bag will also increase the risk of your wheelchair tipping over backwards. We strongly recommend the use of anti-tip wheels.

5. Use

5.1. Transferring from the side

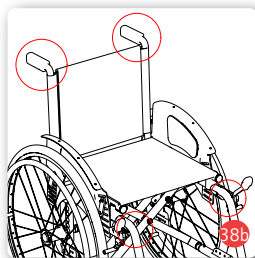
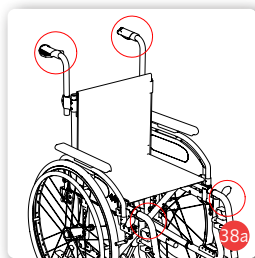
- Make sure that the sides of the wheelchair and the current or future seat surface are as close to each other as possible.
- Secure the wheelchair using the parking brakes (see Chapter 4.7).
- If necessary, swing the armrest backwards or remove the sideguard (see Chapter 4.3).
- Flip both footplates up, swing them to the side or remove them or flip the individual footplate up (see Chapter 4.2).
- Slide sideways to the other seat surface. When doing this sit as far back as possible.
- Move the legrests back into their original position.
- If necessary lock the sideguards back in place.

5.2. Transferring from the front

Never stand on the footplate, there is a
Risk of tipping over!

- Flip the footplates up or remove the legrests (see Chapter 4.2).
- Make sure that the front of the wheelchair and the current or future seat are as close to each other as possible and apply the parking brakes on both sides (see Chapter 4.7).
- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- Now slide onto the seat surface by rotating your body.
- Then hang the legrests back into the initial position and flip the footplates down.

5.3. Going up or down stairs or a high step

Only the frame and the push handles must be used as grip points.

The helpers only grip on the firmly fixed frame parts. The lifting points depend on the model features.

BX 11 (with flip-up footplate (Fig. 38a)

BX 11 S (with swing-away legrests) (Fig. 38b)

In order to climb stairs, the rear helper pulls the wheelchair backwards up the steps by holding onto firmly-mounted handles. The lower helper grips on the front frame tubes and stabilises the wheelchair. By doing this he pushes the drive wheels of the wheelchair onto the steps.

When descending the lower helper brakes the chair by pressing the wheelchair straight into the steps. The upper helper holds the wheelchair by the push handles, keeps it secure and holds it in the right position. The wheelchair should roll down step by step.

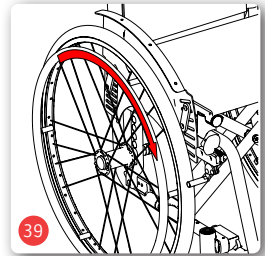
Two attendants must help with obstacles and stairs with more than three steps.

5.4. Driving the wheelchair

- Sit in your B+B wheelchair. Make sure that you sit as far back as possible.
- Movement is achieved using the handrims on the drive wheels. Place your hands on the handrims using your thumbs and bent index fingers (Fig. 39).
- From this position, push the wheelchair with both hands.
- Turning right: hold the right handrim tight and only push with the left one
- Turning left: push in the opposite way
- Turning on the spot: push the handrims in opposite directions at the same time
- Braking: reduce the speed of the handrims with your hands.

Carry out the first test drives carefully, until you have got used to the wheelchair and its driving characteristics. Wheelchairs only have limited security against tipping over and slipping. Take extra care when braking, starting off or turning on slopes. Leaning out of the wheelchair a long way also increases the risk of tipping over. Tipping protection reduces this risk. In order to Negotiate obstacles more easily, the attendant can tip the wheelchair (see Chapter 4.9.1).

6. Technical data

Dimensions and weights

Total length with legrests (mm):	910 - 1010 (BX 11), 1010 - 1120 (BX 11 S)
Total length without legrests (mm):	760 - 870 (BX 11S)
Width (mm):	SW+185 (at 0° camber)
Height (mm):	820 (with back height 300)
Folded Width (mm):	300 (at 0° camber)
Height of the handgrip (mm):	800-950
Lower leg length (mm):	430 - 535 (BX 11) 430 - 510 (BX 11 S)
Footboard angle (°):	+/- 25 (BX 11) +/- 10 (BX 11S)
Knee angle (°):	108 (BX 11) 105 (BX 11S)
Kerb weight (kg):	13,4 - 14,8 (BX 11) 13,8 - 15,0 (BX 11S)
Max. Payload (kg):	135
Seat width (mm):	360 - 520 with 20 mm steps
Seat depth (mm):	380 - 540
Front seat height (mm):	490 - 560 with 10 mm steps
Rear seat height (mm):	420 - 520 with 20 mm steps
Backrest angle (°):	75/81/87/93/99
Backrest height without seat cushion (mm):	300 - 450 with 25 mm steps
Armrest height without seat cushion (mm):	120
Armrest length (mm):	340 (option)
Armrest width (mm):	50 (option)

Drive characteristics


Turning circle (mm)	1200
Static stability (°)	
Forwards:	15
Backwards:	15
sideways:	3.8

Tyres

Drive wheels (inch):	24x1, pneumatic tyres, max.7.5bar
Castors (mm):	150 x 30, PU puncture-proof
Camber (°):	0 - 1

Features

Frame:	Aluminium, painted
Clothes guard:	Active mudguard: Alu removable sideguard: Plastic (ABS)
Seat and back cushion:	Nylon, textile
Armrest:	PU foam (optional)
The dimensions stated are below the manufacturing tolerances of +/- 10 mm.	

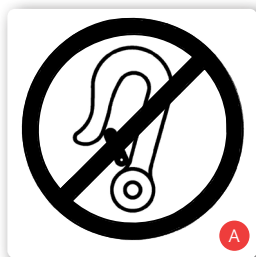7. Transport

7.1. Transporting the wheelchair

In order to make the wheelchair as manageable as possible to transport, remove all detachable parts (legrests, drive wheels, armrests, loose accessories) and fold the wheelchair up (see Chapter 4.1).

7.2. Transportation in vehicles

You can see from the symbol on your product that your wheelchair is not permitted for use as a seat in a vehicle for transporting individuals with impaired mobility (KMP) (Fig. A):

The wheelchair is not crash-tested and must therefore not be used as a seat in a KMP vehicle under any circumstances. There is a risk of death!

8. Maintenance instructions for the user

8.1. Cleaning and disinfection

- Wipe the frame parts of the wheelchair with a damp cloth. If there is heavy contamination, also use a mild cleaner.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush).
- Wash the push handles, armpad, seat and back cushion with a mild cleaner.
- For disinfection use a disinfectant listed by the Association for Applied Hygiene.
- Upholstery parts can be machine washed at 40 ° C. Not machine dry, only drying at room temperature.

Please observe the enclosed overview on care, hygiene and maintenance work

8.2. Checks before driving

8.2.1. Checking the brakes

Each time before setting off, check that the brakes work properly. With the brakes applied, the wheelchair cannot be pushed on dry surfaces. When the brakes are released, the wheelchair runs without scraping noises and good directional stability. If this is not the case, notify your dealer immediately and stop using the wheelchair until it has been serviced!

8.2.2. Checking the tyres

Each time before setting off, check the tyres for damage and that the air pressure is correct (see Chapter 4.8.2).

9. Adjustment work for the specialised retailer

The adjustment work outlined in this chapter affect safety-related criteria of your product and must therefore only be carried out by trained professionals.

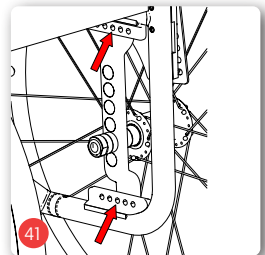
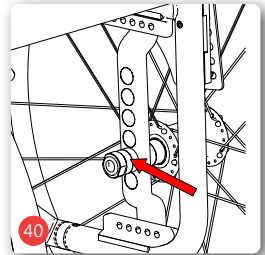
9.1. Centre of gravity, seat angle and seat height

9.1.1. Adjusting the drive wheel

The position of the drive wheel affects the driving characteristics of your wheelchair.

Change the position of the drive wheel.

- Vertical: Adjusting the seat height and angle (Fig. 40).
- Horizontal: Adjusting the centre of gravity position (Fig. 41).


Seat angle

- Horizontal or forward tilted seat surface: fit the quick-release axle bushing in the lowest position. But a seat angle tilted forwards also depends on the position of the front wheel.
- Pronounced seat tilt to the back: fit the quick-release axle holder in the highest position. In this position the seat angle and centre of gravity shift backwards and the wheelchair can tip over backwards more easily. For safety reasons, please use anti-tip wheels!

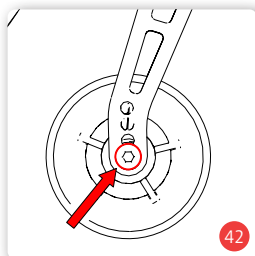By changing the seat angle there is a risk of tipping over and therefore an increased risk of injury.


42

9.1.2. Height of the front wheel

After adjusting the drive wheel position, now check the position of the castors and adjust if necessary.

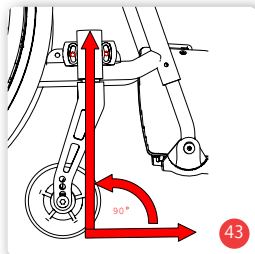- Undo the axle and remove it completely (Fig. 42).
- Position the castor for the required seat height.
- Put the axle back in the castor fork and tighten.
- Adjust the steering axle perpendicular to the floor, in order to achieve directional stability and safe handling (see Chapter 9.1.3).

43

9.1.3. Angle of the steering axle

By changing the position of the drive wheel or front wheel, the angle of the steering axle may change. However this must always be at right angles (90°) to the floor (Fig. 43).

- Release the lock of the castor socket, by undoing both screws on the outside (Fig. 44). Fine adjustment and fixing in a specific position is carried out using the stud screws within castor bushing.
- Put the front wheel into the required position and then re-tighten the fixing screws.

See the assembly instructions for further details.

9.1.4. Wheelbase

The wheelchair's stability and steadiness is improved by extending the wheelbase. Conversely, if the wheelbase is reduced, the manoeuvrability is improved. Reducing the wheelbase is only recommended for experienced riders.

If the wheelbase is changed, then it is essential to adjust the knee lever brake as well!

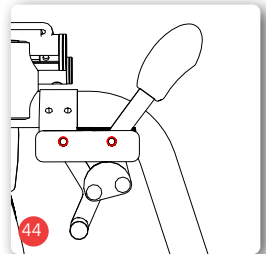
9.2. Adjusting the parking brakes

To adjust the brakes, contact your dealer. Please bear in mind that if the brakes are set incorrectly this can have life-threatening consequences.

If the brakes are not working properly or evenly, contact your dealer. He will test the brakes as follows:

- If there is uneven wear on the tyre tread, or the tread is no longer visible, replace the tyres.
- If the tread is not visible at the brake pins, replace the brake pins.
- Move the braking system to a position that offers a safe braking effect in all driving situations and can be actuated.
- To adjust the gap, undo the two socket screws on the brake bracket (Fig. 44).
- Then push the brake system to the correct position along the bracket.
- Finally re-tighten the two socket screws.
- Check whether the brakes click into place when operating the lever and that they don't snap back.
- Then tighten the screws again and carry out a brake test!


Make sure that both drive wheels have the same air pressure of 7.5 bar.

9.3. Adjusting the drum brakes

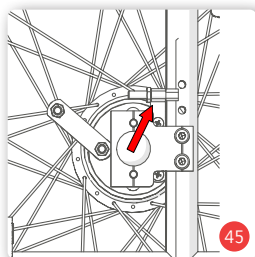To adjust the brakes, contact your dealer. Please bear in mind that if the brakes are set incorrectly this can have life-threatening consequences.

In order to achieve the very best braking performance, adjust the braking force using the adjusting screw. Braking force will be increased by unscrewing the adjusting screw and reduced by screwing it in (Fig. 45).

- Please unscrew the adjusting screw until you hear a scraping noise when the wheel is rotating.
- Then screw the adjusting screw in until the scraping noise stops. The wheel then runs freely.
- When you have finished making the adjustment, the adjusting screw is fixed by tightening the locknut.

9.4. Adjusting the anti-tip wheels

Anti-tip wheels prevent the wheelchair from tipping over backwards.

- Adjust the gap between wheel and floor to approx. 30 - 50 mm: pull the disc back and move the anti-tip wheels along the guide (Fig. 46).

Anti-tip wheels are particularly recommended for inexperienced wheelchair users or amputees.

If the user does not need anti-tip wheels, you can push these to the upper position or pull them out completely (see Chapter 4.9.2).

For safety reasons, always use the anti-tip wheels as a pair!

10. Repairs

Repairs and servicing work must only be carried out by authorised partners (medical supply stores or dealers). To find a partner in your area, please contact our Customer Service on tel.: +49 (0)700/6000 7070.

11. Maintenance information for specialised retailers

Typical wear parts are brakes and tyres. Please contact your dealer for spare parts.

We recommend that you have your dealer check your wheelchair regularly.

If there are faults or defects, pass the wheelchair to your dealer for servicing immediately.

On request, any information and documents necessary for the repair and servicing will be provided to the dealer. A dealer training programme is available.

If you have any questions, B+B Customer Service is available to help you on +49 (0)700/6000 7070.

12. Passing the wheelchair on to another user

The wheelchair is designed for re-use. Before it is passed on to another user, it must be serviced by a dealer and hygienically cleaned. When passing the wheelchair on to another user

please remember to hand over this user manual and the address Of your dealer to the new user.

The wheelchair must be checked in accordance with the maintenance plan and be in Proper technical condition.

13. Storage/shipping

If the wheelchair is to be stored or shipped, all detachable parts which are not fixed, except for the drive wheels, must be removed and packed individually in an appropriate cardboard box. The individually-packed parts can then be packed together in a larger cardboard box. We recommend that you keep the original packing and store it for these purposes, so that it is available when needed. Your wheelchair is then optimally protected against environmental influences during storage or transportation.

14. Disposal

If your wheelchair is no longer in use and needs to be disposed of, contact your dealer.

Check your wheelchair regularly in accordance with the maintenance plan.

If you want to handle the disposal yourself, find out about the regulations in your area from local recycling companies.

15. Warranty

The warranty covers all defects of the wheelchair which are proven to be due to faulty material or manufacturing. In the event of a claim, submit the fully completed warranty certificate.

We provide the legal warranty of 2 years for our products.

For replacement devices and repairs, the warranty period is 1 year.

Please follow the B+B care, warranty, hygiene and maintenance information. Your dealer can provide you with this.

Warranty certificate

Product: BX 11 / BX 11 S

Serial number:*

* (To be filled in by the dealer)

Dealer:

Date and stamp

(Pay attention to our general terms and conditions.)

*Signs are located on the crossbar

Failure to observe the instructions in the User manual as well as not carrying out maintenance work properly as well as, in particular, making technical modifications and additions (attachments) without the agreement of Bischoff & Bischoff GmbH will lead to loss of warranty and the manufacturer's liability.

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