Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

Pyrolino

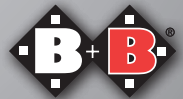
DE

EN

ES

FR

Enjoy mobility.

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Index

1. Preface	38		
1.1. Signs and symbols	38	4.6.2. Front wheel height	52
2. Important safety instructions	39	4.6.3. Angle of the steering wheel axle	53
2.1. General safety instructions	39	4.7. Wheelbase	53
2.2. Anti-tipping wheels	41	4.8. Brake	54
2.3. Travelling in road traffic	41	4.8.1. Activation of the	54
2.4. Purpose	41	4.8.2. Adjusting the locking brake	54
2.5. Indications	41	4.9. Back rest	55
2.6. Contraindications	42	4.9.1. Backrest covering	55
2.7. Declaration of Conformity	42	4.10. Push handles	55
2.8. Responsibility	42	4.11. Wheels and tyres	55
2.9. Service life	42	4.11.1. Driving wheels	56
3. Product and delivery overview	42	4.11.2. Steering wheels	56
3.1. Checking the delivery	43	4.12. Quick-release axles	56
3.2. Type plate and serial number	43	4.13. Tipping aid	56
3.3. Scope of supply	43	4.14. Anti-tippers/anti-tip wheels	56
3.4. Overview	44	4.15. Safety belt	57
4. Assembly/adjustments	46	5. Use	57
4.1. Folding and unfolding	46	5.1. Getting in and out from the side	57
4.2. Leg rests with footrest and calf pad	47	5.2. Getting in and out from the front	58
4.2.1. Fitting the leg rests	47	5.3. Negotiating stairs and high steps	58
4.2.2. Swinging the leg rests out of the way	47	5.4. Moving the wheelchair	59
4.2.3. Removing the leg rests	48	6. Technical data	62
4.2.4. Adjusting the leg rest	48	7. Transport	62
4.2.5. Adjusting the foot plate	48	7.1. Transporting the wheelchair	62
4.2.6. Leg rest with calf pad (optional)	48	7.2. Seat in disabled vehicle	62
4.2.7. Adjusting the leg rest with calf pad.	49	8. Maintenance tips for users	63
4.2.8. Adjusting the calf pad	49	8.1. Cleaning and disinfection	63
4.2.9. Adjusting the knee angle.	49	9. Check before use	64
4.3. Armrests/side parts	50	9.1. Brake Inspection	64
4.3.1. Armrest height	50	9.2. Checking the tyres	64
4.4. Seat height	50	10. Maintenance tips for retailers	64
4.5. Seat angle	50	11. Use by others	65
4.6. Adjusting the wheels	52	12. Storage / shipping	66
4.6.1. Driving wheel height	52	13. Disposal	66
		14. Warranty	68

1. Preface

Dear user,

Your purchase of a Bischoff & Bischoff Pyrolino folding wheelchair means you are getting a top-quality German product that sets new standards in terms of versatility. Pyrolino is intended both for indoor and outdoor use.

This operating manual is aimed primarily at the parents and guardians of the children who use this wheelchair.

This operating manual should help you become more familiar with the operation and use of the wheelchair. Please read and follow the operating manual carefully before operating your new wheelchair for the first time. The operating manual is an integral and necessary component of the wheelchair. Keep the operating manual where you can refer to it, and if you pass the wheelchair on to someone else, give the manual to the new owner.

For users with visual impairment, this document is available as a PDF file on our internet page www.bischoff-bischoff.com.

1.1. Signs and symbols

Important! Indicates particularly useful information in the specific context.

Caution! Indicates information that is specifically relevant to safety. Follow the operating manual!

2. Important safety instructions

2.1. General safety instructions

- In order to avoid toppling and dangerous situations, the handling of the new wheelchair should be practised in a flat, easily manageable area. We recommend having someone accompany you in this situation.
- Before using the wheelchair, please check whether all fitted parts are properly attached.
- Never lift the wheelchair by parts that are not securely attached (armrests or leg rests).
- Make sure that the quick-release axles are seated properly in the axle holders – otherwise there is a risk of injury.
- Please note that the risk of tipping to the side and tipping backwards may increase due to shifts in equilibrium caused by body movements or loads on the wheelchair.
- The wheelchair must not be misused by transporting more than one person. Please ensure that you respect the intended purpose of the product.
- Please note there is a risk of becoming trapped between moving parts when adjusting, using and maintaining the wheelchair.
- Someone should be asked to accompany you when negotiating obstacles (steps, etc.). Use the tip assist (see overview of the wheelchair) to tip the wheelchair in an appropriate way.
- Steps and larger obstacles may only be negotiated with the help of at least two companions. Use lifting points 1 and 2 for this (see 5.3.). If facilities such as access ramps, lifts or step climbing aids are available, these should be used.
- For journeys on a slope/incline, it must be noted that, depending on the axis position that has been set, there is a risk of tipping even if the slope/incline is only a few degrees. Use of anti-tipping wheels is recommended.
- Avoid driving into an obstacle without braking (e.g. steps, kerbs) or "jumping down" from one level to another.
- Avoid journeys on uneven, loose surfaces.

- When getting into or out of the wheelchair, apply both locking brakes.
- Always follow the road traffic regulations when on the road.
- The passive lighting (reflectors) on your wheelchair must always be visible to other road users.
- The locking brake must not be used to brake when moving.
- When braking from fast speeds or on long downward slopes by using the push rims, fingers and palms get hot. Attention, danger of burning!
- We recommend the use of leather gloves outdoors. These improve your grip and simultaneously protect your fingers and palms from dirt and injury.
- Please note that upholstery parts which are exposed to direct solar radiation heat up and can cause injury to the skin on contact. You should therefore cover these parts or protect the wheelchair from solar radiation.
- Also note that frames and upholstery parts can cool off greatly in the winter. If possible, do not store the wheelchair outside in case of cold weather.
- When transferring between wheelchair and bed, please always ensure that:
 - both locking brakes are applied.
 - the side part on the side for getting in/out is swung back.

2.2. Anti-tipping wheels

The anti-tipping wheels prevent the wheelchair from tipping backwards. Please note that using the wheelchair with the anti-tipping wheels is only possible to a limited extent on areas with stairs, lifting platforms and ramps. Make sure that there is an adequate distance at the top, bottom and sides.

2.3. Travelling in road traffic

The wheelchair is designed for indoor and outdoor use. Please be aware that you are travelling on public roads and must comply with the road traffic regulations. Do not endanger other participants through careless driving, particularly on footpaths.

2.4. Purpose

The wheelchair is designed solely to increase mobility and to transport people/children who have difficulty walking, with the indications mentioned. The user load is a maximum of 100 kg.

2.5. Indications

- Walking disability or serious difficulty in walking as a result of:
- Paralysis
- Limb loss
- Limb defect/deformity
- Joint contractures (not on both arms)

2.6. Contraindications

It is not appropriate to use the wheelchair when a patient has:

- Disorders of perception
- Severe equilibrium disorders
- Limb loss of limb on both arms
- Joint contractures or joint injury on both arms
- Inability to sit
- Reduced or inadequate vision

2.7. Declaration of Conformity

B+B, as the manufacturer of the product, declares that the Pyrolino wheelchair is in complete compliance with the requirements of the directive 93/42/EEC.

2.8. Responsibility

We are able to provide warranty cover only if the product is used in accordance with the specified conditions and for the intended purposes, if modifications, extensions, repairs and maintenance work are carried out only by persons we have authorised to do so and if the wheelchair is used in accordance with all instructions for use.

2.9. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life of the wheelchair is up to 5 years. Beyond this time, the wheelchair can continue to be used provided that it is in a safe condition.

3. Product and delivery overview

3.1. Checking the delivery

All Bischoff & Bischoff GmbH products are subjected to a systematic final in-house inspection and provided with the CE mark.

The wheelchair is supplied in a special carton with factory settings. After unpacking, please retain the carton if possible. This can be used to store or return the product at a later stage.

Delivery and training are usually carried out by the qualified medical supplies retailer.

If shipped by rail or a forwarding agent, the goods are to be checked with regard to transportation damage at once in the presence of the delivery company.

Check that the contents are complete and intact. In the event of irregularities and damage, contact our customer service department by telephoning: +49 (0)700/6000 7070.

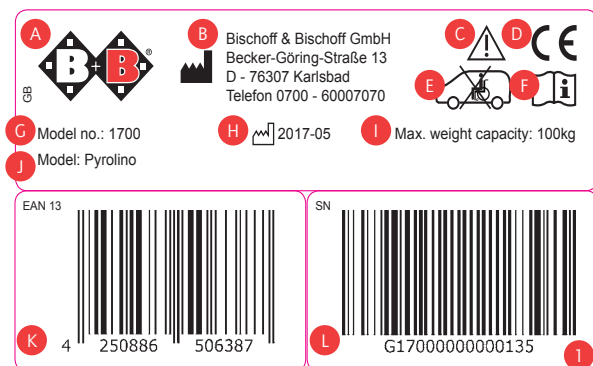
3.2. Type plate and serial number

The type plate (Fig. 1) and serial number are located on the cross strut.

3.3. Scope of supply


- When you receive the product, please check immediately that the contents are complete. The contents comprise:
- Outer packaging
- Wheelchair pre-assembled
- Instruction manual


3.4. Overview

- A** Manufacturer's logo
- B** Manufacturer's information
- C** Caution! - Follow instructions for use
- D** CE mark
- E** Follow instructions regarding the securing of the product in the vehicle
- F** Important - Follow instructions for use
- G** Model number
- H** Date of manufacture
- I** Maximum load capacity
- J** Model designation
- K** EAN-13 code
- L** Serial number

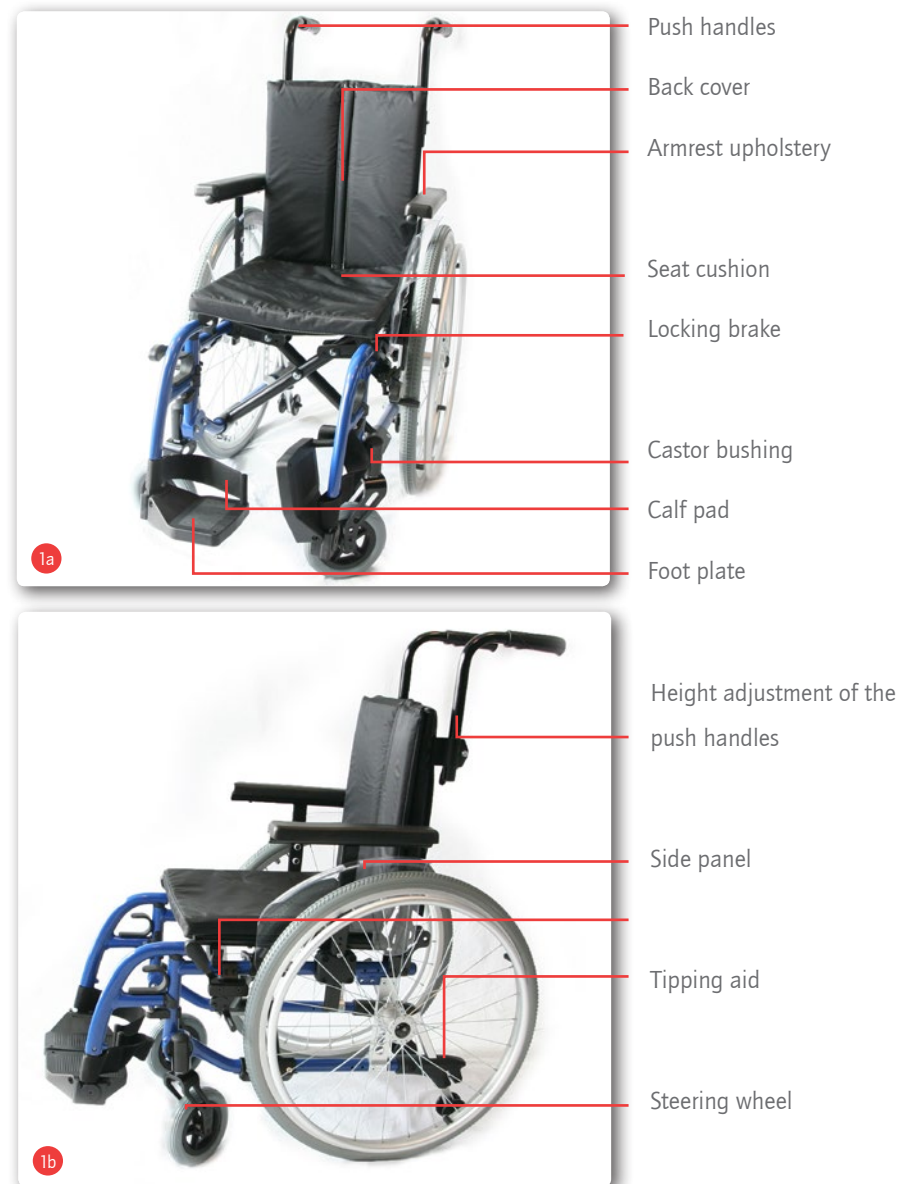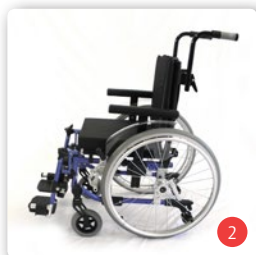Pyrolino model

4. Assembly/adjustments

With a little technical skill, you can carry out the adjustments on your wheelchair yourself. To do this, you need an Allen wrench set with sizes from 3 mm to 6 mm and ring spanners SW10, SW13. If you have any questions, you can contact your specialist dealer.

Please be careful with your fingers when making adjustments. Risk of becoming trapped!

4.1. Folding and unfolding

Unfolding

Place your wheelchair next to you. Tip it towards you until the load is taken off one driving wheel (Fig. 3). Now press on the seat tube nearest to you with the flat of your hand to unfold the wheelchair (Fig. 4).

When unfolding, please make sure your fingers remain on the seat belt. Otherwise there is a risk of becoming trapped.

Folding

Place your B+B wheelchair next to you. Tip it by pulling the backrest towards you until the load is taken off the opposite driving wheel. At the same time, pull the seat cushion upwards until the chair is completely folded (Fig. 5).

To avoid injuries, do not hold the wheelchair at the armrest while tilting. If leg rests are fitted, the foot plates should be folded up first.

4.2. Leg rests with footrest and calf pad

Both leg rests are supplied with the foot plates and heel straps fitted. The foot plates on both leg rests can be folded upwards in the sitting position.

4.2.1. Fitting the leg rests

- Hold the upper part of the leg rest so that it points outwards (Fig 7). Insert the left rest into the hole in the frame with the guide profile (Fig. 6).
- The leg rest is now simply turned forward, which locks it automatically into place.

Check that the foot rests are firmly attached before setting off.

4.2.2. Swinging the leg rests out of the way

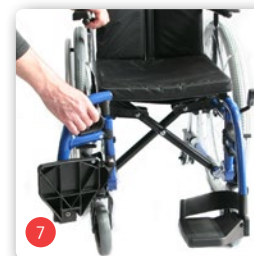
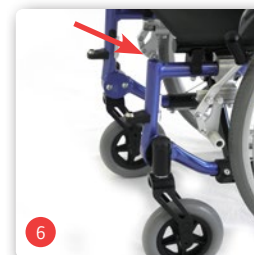
To make it as easy as possible to get into or out of the B+B folding wheelchair, if necessary you can simply swing the leg rests out of the way, either outwards or inwards, without having to remove them.

Pull the handle. This will open the locking mechanism (Fig. 7).

Now you can swing the leg rest away, outwards or inwards (Fig. 8).

If necessary, fold the foot plates upwards first.

To return to the original position, swing the leg rest forwards until it locks into place.

When standing up from the wheelchair, you must not stand on the foot plates under any circumstances

4.2.3. Removing the leg rests

To transport your wheelchair as compactly as possible, you can completely remove the leg rests along with the foot plates (Fig. 9).

- Pull the handle in order to open the locking mechanism.
- Swing the leg rest outwards 90° and pull it upwards (Fig. 9).

4.2.4. Adjusting the leg rest

The lower leg length can be adjusted in six levels. (Fig. 10). Open the Velcro on the heel straps. Loosen the hexagon socket screw (4mm) and slide the tube to the desired length. Ensure that the complete number is visible in the window. Retighten the hexagon socket screw. Ensure that both leg rests are adjusted to the same level

4.2.5. Adjusting the foot plate

The angle of the foot plates can be adjusted by simply unscrewing the hexagon socket screw. Adjust the foot plate to the desired position by twisting the sprocket (Fig. 10). Secure the hexagon socket screw again

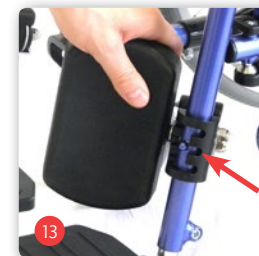
4.2.6. Leg rest with calf pad (optional)

As an alternative you have the option to get a leg rest with foot plates and calf pads.

4.2.7. Adjusting the leg rest with calf pad.

The length of the leg rest can be individually adapted. Loosen the Allen screw (Fig. 12) and slowly pull out the tube to which the foot rest is attached in a downwards direction. The adjustment is variable. The knurled area marks the maximum length. The knurling must not be visible. Repeat the same process with the other side. Ensure that both lengths are adjusted equally.

4.2.8. Adjusting the calf pad

Height

As standard practice, the height of the calf pad can be adjusted in three stages (Fig. 13). To do this, rotate the calf pad out of the groove, push it along the tube and rotate into back into the groove.

Depth

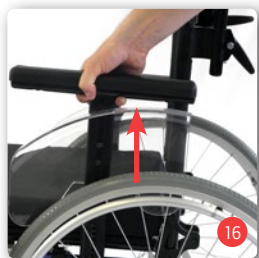
You can also adjust the depth of the calf pads. Loosen the rear screw, push the pad into the desired position and tighten the screw again. (see Fig. 13).

4.2.9. Adjusting the knee angle.

The knee angle can be adjusted in a wide range, up to a horizontal position.

- Loosen the knurled screw (Fig. 15).
- Move the leg rest into the desired position by raising or lowering the foot plate support tube.
- Tighten the knurled screw again

4.3. Armrests/side parts

The side parts are securely fixed on the armrests. To make it easier to get into or out of the wheelchair, simply pull out the armrests with the side parts upwards (Fig. 16).

Do not use the side parts to carry the wheelchair!

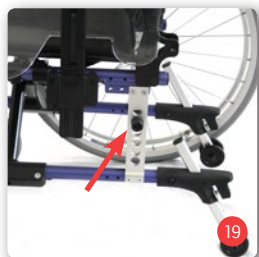
4.3.1. Armrest height

You can adjust the armrest height. To do this, the sliding block must be pushed along the armrest rail. Loosen the Allen screw (Fig. 17) with a size 4 mm Allen wrench. Move the armrest to the desired position and tighten the Allen screw again. Repeat this procedure on the other side. Please make sure that both armrests are always adjusted to the same height.

4.4. Seat height

The seat height can be changed by simultaneously adjusting the driving wheel and the front wheel (see item 4.6). In the process ensure that the driving and front wheels are adjusted in the same direction and by the same value.

4.5. Seat angle

If you incline the seat of the wheelchair backwards, this means you will be seated more deeply and firmly in the chair. To lean the seat backwards, you have to move the quick-release axles (see driving wheels) to a higher position. If you want to incline the seat forwards, fix the quick-release axle in a lower position. The reverse is true of the front wheels. If you attach the front wheel lower in the fork, the seat will be inclined backward. If you attach the front wheel higher in the

fork, this results in a negative seat inclination towards the front.

4.6. Adjusting the wheels

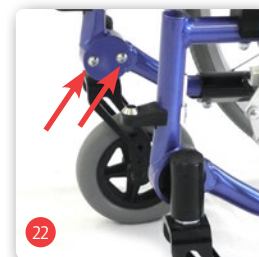
4.6.1. Driving wheel height

Remove the driving wheel (Fig. 18) to make work easier. To adjust the height of the driving wheel and therefore to raise or lower the entire wheelchair, you must first remove the mounting of the driving wheels. You can use an Allen wrench to loosen the fastening nuts (see Fig. 19) on the axle mounting.

Now you can remove the mounting and attach it at the desired height. Tighten the nut again and attach the driving wheel. Make sure that the driving wheels are mounted at the same height.


When the driving wheel height is changed, the locking brake must be readjusted.

4.6.2. Front wheel height

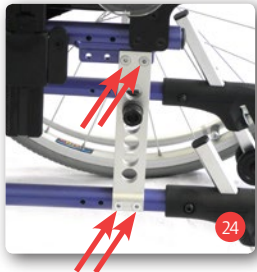The front wheel height can be adjusted in three stages. Loosen the Allen screw (see Fig. 20) and pull out the wheel axle. Attach the front wheel at the desired height and tighten the screw again. Make sure that the front wheels are mounted at the same height.

4.6.3. Angle of the steering wheel axle

By changing the height of the driving wheel or front wheel, the angle of the steering wheel axle can also be altered. However, this should always be at the correct angle (90°) to the floor (Fig. 21).

First, loosen the locking mechanism by loosening both Allen screws on the inside (Fig. 22). You can adjust the angle using the two setscrews on the bottom (see Fig. 23).

Then tighten the two fixing screws on the inside again (Fig. 22).

4.7. Wheelbase

Caution, risk of tipping! If you adjust the driving wheel forwards, this significantly increases the risk of tipping. In this case, anti-tipping wheels are necessary. Check the settings first with a helper. Only use settings that you feel safe with.

Extending the wheelbase increases the stability and equilibrium of the wheelchair.

If the wheelbase is reduced, manoeuvrability is increased. Please note, a reduced wheelbase is only advised for practised drivers.

To adjust the wheelbase, loosen the four fixing screws that are used to attach the wheel suspension (Fig. 24). The wheelbase can be changed in three stages (Fig. 25).

Subsequently, attach the wheel suspension again in the recesses provided for this purpose. Make sure that all screws are firmly tightened. A wheelbase extension can also be used to raise the wheelbase.

When the wheelbase is changed, it is absolutely essential to adjust the locking brake afterwards.

4.8. Brake

4.8.1. Activation of the locking brake (compact brake)

To lock the brake, simply push the brake lever forwards until it clicks into place (Fig. 26). To release the locking brake again, simply pull the brake lever backwards (Fig. 27).

4.8.2. Adjusting the locking brake

If you notice that the available braking effect is no longer sufficient or is uneven, proceed as follows:

- Check the tyre tread depth. Check also that the friction is even. If the tread depth in the middle of the tyre is less than 1 mm, the tyre must be changed. If you have the necessary technical knowledge, you can change the tyre yourself. If you do not think you can do this yourself, your mobility dealer will be able to help you.
- Now check the adjustment of the brake. This must be set so that the gap between the surface of the wheel and the braking bolt is 15 mm with the brake off (Fig. 27). The braking bolt must be aligned horizontally so that it is parallel to the ground surface.
- After loosening the two lens head screws with hexagon socket (5 mm) on the brake mount (Fig. 28), you can position the brake correctly.
- The compact brakes only operate on one driving wheel each.
- You can perform a fine adjustment of the brakes on the two Allen screws (Fig. 29).
- With pneumatic tyres, always ensure that both driving wheels have the same tyre pressure to achieve good directional stability. Once the adjustment process is completed, firmly tighten the loosened screw again and perform a brake test.

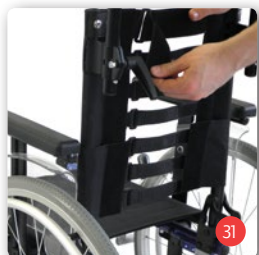
4.9. Back rest

The hardness of the backrest covering can be adjusted to allow for comfortable seating.

4.9.1. Backrest covering

The tighter you adjust the backrest covering, the harder the backrest will feel. First remove the backrest padding that has been attached with a hook-and-loop fastener (Fig. 30). Then you can pull the individual straps so they are tighter or less tight (Fig. 31). To relieve certain areas of your back, do not tighten the straps there as much. Now the back cushion needs to be attached with the hook-and-loop fastener again.

4.10. Push handles

To make it possible for a companion to push the wheelchair comfortably, the height of the push handles is adjustable. Loosen the lever screw (Fig. 32) on the push handle. Pull the tube with the push handle to the desired height (Fig. 33). Then tighten the lever screw again to lock the push handle.

4.11. Wheels and tyres

The spoked wheels are attached with a quick-release axle on the chassis (Fig. 36). The Pyrolino has quick-release axles as standard (quick release axles, see section 4.12).

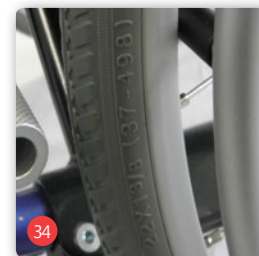
Check that the wheels are securely seated each time they are fitted.

4.11.1. Driving wheels

PU tyres (puncture-proof) with a size of 22" are mounted on your Pyrolino as standard (Fig. 34). Pneumatic tyres are available as an option. If they become worn or defective, the authorised mobility dealer should be asked to perform the repair.


To ensure the optimum operating safety of the brakes or wheelchair, there must be a tyre pressure of 300-400 kPa (34 bar) with pneumatic tyres.

4.11.2. Steering wheels

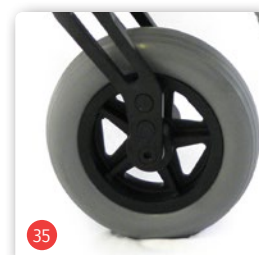
PU tyres (puncture-proof) with a size of 8" are mounted on your Pyrolino as standard (Fig. 35). Pneumatic tyres are available as an option. If they become worn or defective, the qualified mobility dealer should be asked to perform the repair. To ensure the optimum operating safety of the wheelchair, there must be a tyre pressure of 200-250 kPa (22.5 bar) for swivel-wheels with pneumatic tyres.

4.12. Quick-release axles

To make the transport of your wheelchair considerably easier, the wheels are equipped with quick-release axles as standard:

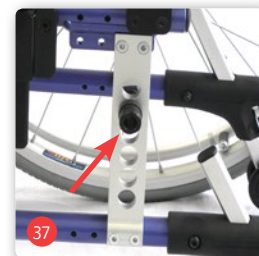- Press the locking button of the quick-release axle with your finger and pull off the wheel (Fig. 36).
- When refitting the wheel, simply push the wheel back onto the mount (Fig. 37). At the same time, press in the locking button as well.

When attaching, make sure that the quick-release axle is inserted right up to the stop. After attaching, check that the wheel is securely fixed by pulling it towards you without operating the locking button of the quick-release axle.

4.13. Tipping aid

The tipping aid can be used to make it easier for a companion to negotiate obstacles such as kerbs. Hold the wheelchair firmly at the push handles with both hands. Then press one of the tip assists with one foot (Fig. 38), while at the same time pushing downwards with your hands.

4.14. Anti-tippers/anti-tip wheels

Always use the anti-tipper as a pair.

To adjust the anti-tipper's distance to the floor, please proceed as follows:

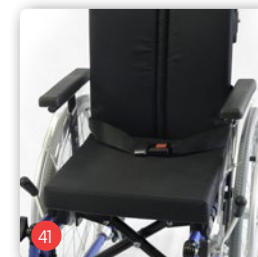
- Pull back the cup washer (Fig. 39).
- Move the anti-tipper into the desired position (Fig. 40).
- Release the cup washer to lock the anti-tipper.
- Please note that some ground clearance is necessary to negotiate obstacles such as kerb edges. The various adjustment options allow you to adjust the anti-tippers to the conditions individually.

The anti-tippers can also be used as a tipping aid when anti-tipping wheels are raised.

4.15. Safety belt

If required, a safety belt is available as an accessory. This is fitted onto the side frame of the wheelchair with the side part either temporarily removed or swung away (Fig. 41). Take the open end of the belt once around the side frame tube and then fix it with the belt buckle. You also use the belt buckle to adjust the length of the belt. Adjust the belt length so that the belt is tight, but does not apply pressure.

5. Use

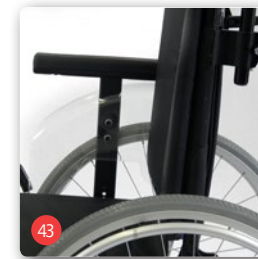
Please be careful of your fingers during use. Risk of becoming trapped!

5.1. Getting in and out from the side

- Move the wheelchair and the current or future seat so they are arranged side by side and as close to each other as possible.
- If the child wants to move from another wheelchair or indoor frame, secure this by activating the locking brakes.
- Secure the Pyrolino against rolling away accidentally by activating the locking brake (Fig. 42).
- Remove the side part with the armrest (Fig. 43).
- Fold the foot plates to the side or remove the leg rests (Fig. 44).
- Now let the child slide laterally onto the other seat.
- Make sure that the child sits as far back as possible in the seat.
- Finally, return the leg rests to their original position. And re-insert the side parts if necessary.

5.2. Getting in and out from the front

- Swing the foot plates upwards or remove the leg rests (Fig. 45).
- Bring the Pyrolino and the current or future seating area so that their respective fronts are as close to each other as possible and brake the Pyrolino (Fig. 46).
- If the child wants to move from another wheelchair or indoor frame, secure this by activating the locking brakes.
- Now let the child slide onto the seat by turning its body.
- Finally, return the leg rests to their original position and swing the foot plates downwards

Never climb on the footrests!
Risk of tipping!

5.3. Negotiating stairs and high steps

In case of obstacles and stairs with more than three steps, help from two companions is needed.

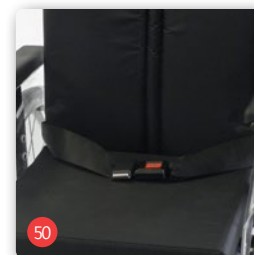
- Helpers must only take hold of securely fitted parts of the frame. The lifting points are represented depending on the model equipment (Fig. 47).
- When going up stairs, the helper behind the wheelchair pulls the wheelchair over the step holding it by the securely fitted handles (Fig. 48).
- The helper who is below takes hold of the front frame tubes and stabilises the position. At the same time, they push the rear wheels of the wheelchair against the steps.

- When going down stairs, the helper who is below brakes by pushing the wheelchair right against the steps. The helper who is above holds the wheelchair by the push handles, secures it and holds it in the right position. The wheelchair should roll down step by step.

The frame and the push handles areare the only parts where the wheelchair can be gripped to transport it. Under no circumstances should armrests be used to lift the wheelchair

5.4. Moving the wheelchair

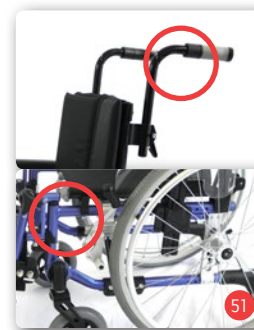
Seat the child in the B+B wheelchair. Make sure that the child is sitting as far back in the seat as possible.

Moving and braking are achieved via the push rims on the drive wheels (Fig. 49). Make sure that the first attempts at movement are carried out carefully, until the child has become accustomed to the wheelchair and its handling.

Wheelchairs have only limited anti-tip and anti-slip security. Particular care is advised when braking, starting or when turning on gradients. Leaning out too far from the wheelchair increases the risk of tipping over. For greater safety, all wheelchairs can be equipped with a safety belt (Fig. 50) (not standard).

Please note that hanging bags and rucksacks on the backrest significantly increases the risk of tipping over backwards

The locking brake must be activated before getting into or out of the wheelchair (Fig. 46).

Journeys outside the residence should only be carried out with sufficient driving experience. If possible, someone should accompany the child. To make it easier to negotiate obstacles, the companion must position the wheelchair at a right angle to the obstacle by pressing the tip assist with the foot.

According to the German traffic regulations (STVZO) dated 1/1/1981, manually operated wheelchairs travelling on public roads (including pavements) must have permanently attached illumination.

This consists of (passive):

- two red reflectors in the back
- two yellow reflectors on the side (on both sides).

6. Technical data

Dimensions

Total length with leg rests:	915 mm
Overall width:	540 mm
Height:	825 - 955
Folded (WxHxL):	320 x 825 x 955 mm
Lower leg length, standard:	280 - 360 mm without seat cushion 340 - 420 mm with seat cushion
Lower leg length optional:	420-480 mm
Seat width:	330 / 350 mm
Seat depth:	370 mm
Seat height (basic setting):	440 mm 490 mm with seat cushion
Seat angle (basic setting):	3°
Backrest angle:	90°
Backrest height:	420 mm
Distance of armrest to seat surface:	165-235 mm with seat cushion
Length of the armrest measured from the backrest:	260 mm
Horizontal position of rear axle, measured from the backrest:	-15 mm

Dimensions

Knee angle:	90 ° (to 180°)
Wheel camber:	0°

Weights

Max. load (kg):	100 kg
total weight:	14,4 kg with seat cushion
Weight of the heaviest part:	8,44 kg

Basic adjustments

Front wheels:	lowest position
Driving wheels:	second position from the top
Wheelbase:	rearmost position

Driving characteristics

Turning radius:	620 mm
-----------------	--------

Ambient conditions

Temperature :	max. 35°C
Air humidity:	max 80%

Storage conditions

Temperature (°C):	40 °C
Air humidity (%):	max 80%

Tyres

Powered wheels:	558,8 mm
Steered wheels:	152,4 mm

Equipment

Folding mechanism:	Single crossbrace
Side parts:	Height adjustable
Leg rest:	Can be removed/swung away, adjustable lower leg length, can be adjusted horizontally
Brakes:	Knee lever tyre brake
Driving wheels axles	Quick-release axle as standard
Frame:	Lacquered aluminium
Attachments:	Lacquered aluminium
Seat cushion and backrest padding:	Nylon / foam
Armrest:	EVA
Flame retardant:	according to EN 1021-1/2

7. Transport

7.1. Transporting the wheelchair

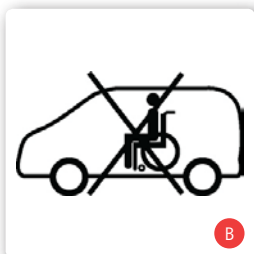
To transport the wheelchair as convenient as possible, all detachable parts (footrests, wheels, accessories) must be removed and the wheelchair folded.

7.2. Seat in disabled vehicle

You can determine whether or not your wheelchair is approved for use as a seat in a vehicle for transporting disabled people by referring to the symbols on the CE sticker on the product:

Wheelchair suitable for use as a seat in a vehicle for transporting disabled people (fig. A)

Wheelchair not suitable for use as a seat in a vehicle for transporting disabled people (fig. B)

All products approved by B+B for use as seats in vehicles are tested in accordance with ISO 7176-19. The tests in accordance with ISO 7176-19 were performed using restraint systems from AMF-Bruns. The correct mounting of the restraint system is a prerequisite for the safe transportation of the products in a vehicle for transporting disabled people and the basis for the approval.

Fixation points on the restraint system (fig. C)

The restraint systems from AMF-Bruns provide the ideal points for the provision of restraint force for the purpose of securing the products in the vehicle for transporting disabled people (known by the German term „Kraftknoten“, which is defined in the standard DIN 75078-2). For the purpose of providing restraint in the vehicle for transporting disabled people (fig. C), the products must only be secured at these specific points („Kraftknoten“ points). If possible, use a permanently installed car seat and its belt system. Stow the wheelchair in the luggage space. The person responsible for the transportation should be

familiar with the current regulations for transportation as well as with the method of operating the vehicle for transporting disabled people and the restraint system.

Anchored 4-point restraint systems must be used to secure the wheelchair in the vehicle. If you have any queries about securing your product in a vehicle for transporting disabled people, please contact the B+B customer service department on +49 (0)700/60007070.

8. Maintenance tips for users

8.1. Cleaning and disinfection

- The frame and the paneling of the wheelchair can be cleaned with a moist cloth. You can use a mild detergent for more stubborn dirt
- The wheels can be cleaned with a wet synthetic fiber brush (do not use a wire brush!)
- Push handles, arm pads, and seat and back pads can be washed off. Use mild detergent.
- Use a disinfectant listed by VAH for disinfection.

Observe our included overview of the most important care, hygiene and maintenance work.

9. Check before use

9.1. Brake Inspection

Make sure your brakes are operating properly before each use. It should not be possible to push the wheelchair away while the brakes are activated.

The wheelchair should be easy to drive without grinding noises and with good direction stability when the brakes are released.

If you notice unusual behaviour when braking, inform your qualified mobility dealer immediately and do not use the wheelchair.

9.2. Checking the tyres

Before embarking on any journey, check that the tyres are in good condition.

10. Maintenance tips for retailers

We recommend having the wheelchair inspected by your qualified mobility dealer at least once a year. If faults or defects at the wheelchair are found, it must be returned to the dealer for repair immediately. The dealer can receive the required information and documentation for repair and service on request. A training program is also available for dealers.

Repair and replacement of parts are carried out by the qualified mobility dealer.

B+B customer service is available at +49 (0)700/60007070.

The wheelchair must be inspected in accordance to the service plan and must be in good condition.

11. Use by others

The children's wheelchair Pyrolino is appropriate for use by others. For use by others, the wheelchair must first be thoroughly cleaned and disinfected according to the accompanying care and hygiene instructions.

The product shall get checked by an qualified mobility dealer for general condition, wear and shall get repaired if necessary.

12. Storage / shipping

If the wheelchair is to be stored or shipped, all attachable and unsecured parts, apart from the drive wheels, must be removed and packed individually in appropriate cartons. The individually packed parts can then be packaged together in a larger carton. We recommend to keep the original carton and store it for those purposes so that it is available if required. Your wheelchair will have an optimum protection against environmental influences during storage and transport.

13. Disposal

If the wheelchair is no longer used and you wish to dispose it, please contact your dealer.

If you wish to dispose the wheelchair yourself, please contact a local recycling company for the disposal regulations of your local jurisdiction.

14. Warranty

The warranty covers all defects of the wheelchair that are demonstrably due to material or manufacturing defects. Any claim must be accompanied by the completely filled-out warranty certificate.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of B+B will generally lead to forfeiture of warranty and product liability claims.

Our products come with a statutory guarantee of 2 years. For replacement devices and repairs, a warranty period of 1 year applies.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

Warranty-Certificate

Model: Pyrolino

Model number:

Serial number:*

* (to be filled in by authorized dealer)

Dealer:

Date and stamp

(Please note our general business terms and conditions))

*The serial number is located on the side frame.

Tip!

In spite of careful research and editing, incorrect information may have crept into our operating manuals. The current version of each of the operating manuals can be found in the download area of our website: www.bischoff-bischoff.com

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B+B France S.A.R.L.