
Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

S-Eco 2
S-Eco 300
S-Eco 300 XL
Pyro Start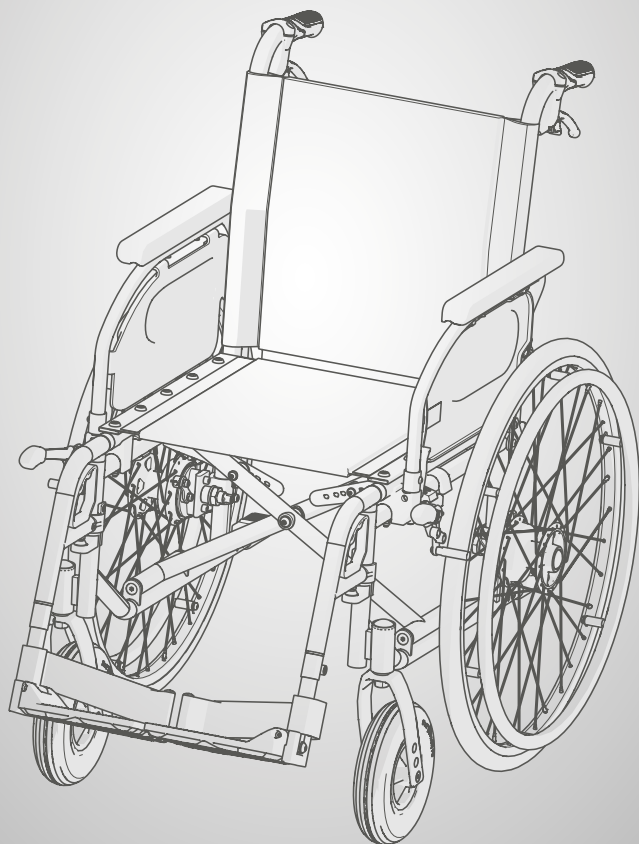


DE

EN

ES

FR

Enjoy mobility.

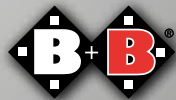Inhalt

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S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

Contents

1. Introduction	44
1.1. Labels and symbols	44
2. Important safety information	45
2.1. General safety information	45
2.2. Risk of tipping over	47
2.3. Safety in vehicles	47
2.4. Use on roads	47
2.5. Intended use	48
2.6. Indications	48
2.7. Contraindications	48
2.8. Declaration of conformity	49
2.9. Liability	49
2.10. Service life	49
3. Product and scope of delivery	50
3.1. Checking the delivered goods	50
3.2. Scope of delivery	50
3.3. Product's identification label and serial number	51
3.4. Overview	52
4. Assembly/Adjustments	53
4.1. Folding and unfolding	53
4.2. Legrests	54
4.2.1. Swing-away legrests	54
4.2.2. Adjusting the legrests	55
4.2.3. Horizontally-adjustable legrests (optional)	55
4.3. Amputee legrests (optional)	55
4.4. Sideguards	56
4.4.1. Universal sideguard	56
4.4.2. Height and depth-adjustable sideguards (optional)	56
4.5. Brakes	57
4.5.1. Operating the parking brake	57
4.5.2. Adjusting the parking brakes	58
4.5.3. Operating the parking brake with the brake lever extension	59

4.6. One-hand brake (optional)	59
4.6.1. Drum brake (optional)	59
4.7. Wheels and tyres	60
4.7.1. Quick-release axle	60
4.7.2. Drive wheels	61
4.7.3. Castors	61
4.8. Transit wheels (optional)	62
4.9. Wheelbase extension (optional)	62
4.10. Tipping aids	62
4.11. Anti-tip wheels (optional)	63
4.12. Headrest (optional)	63
4.13. Adjustable backrest (optional)	63
4.14. One-hand operation (optional)	63
4.15. Reflectors	63
5. Accessories	64
5.1. Lap belt (optional)	64
5.2. Therapy table (optional)	64
5.3. Togo (optional)	64
5.4. Crutch holder (optional)	64
5.5. Spokeguard (optional)	64
5.6. Handrim cover (optional)	65
5.7. Infusion & oxygen bottle holder (optional)	65
5.8. Fixing a bag (optional)	65
6. Use	66
6.1. Side transfer	66
6.2. Frontal transfer	66
6.3. Going up or down stairs or a high obstacle	67
6.4. Driving the wheelchair	68
7. Technical data	70
8. Transport	72
8.1. Transporting the wheelchair	72
8.2. Transportation in vehicles	72

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

9.	Maintenance instructions for the user	73
9.1.	Cleaning and disinfection	73
9.2.	Checks before driving	73
9.2.1.	Checking the brakes	73
9.2.2.	Checking the tyres	74
10.	Repairs	74
11.	Maintenance information	74
12.	Passing the wheelchair on to another user	74
13.	Storage/shipping	75
14.	Disposal	75
15.	Warranty	76

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

EN

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

1. Introduction

Dear user,

You have chosen a high-quality wheelchair from B+B. Thank you for trusting in us.

This user manual contains all the information necessary to operate the S-Eco 2 / S-Eco 300 / Pyro Start. / S-Eco 300 XL wheelchairs.

Your wheelchair was designed for both indoor and outdoor use.

Before using your new wheelchair for the first time, please read and follow the information in this User manual carefully. This is an integral and necessary part of the wheelchair. Keep the User manual within easy reach and pass it on when transferring the wheelchair.

For users who are visually impaired, this document is available as a pdf file on our website www.bischoff-bischoff.com.

Note!

Despite thorough research and editing, incorrect information may occur in our User manual. We reserve the right to make technical modifications. Images may show accessories. You can find the current version of the User manuals in the download area of our website:

www.bischoff-bischoff.com

1.1. Labels and symbols

Important! Indicates particularly useful information in the relevant context.

Attention! Means particular safety-related Notes. Observe the User manual!

2. Important safety information

2.1. General safety information

- In order to prevent falls and dangerous situations, practise with your new wheelchair on level, clear ground first. An attendant is recommended.
- Before using the wheelchair check that all attached parts are properly fixed.
- Only lift the wheelchair using firmly installed parts (see Chapter 6.3).
- The risk of tipping over to any side increases with changes in balance through body movements or loading the wheelchair.
- The wheelchair must not be used to transport several individuals or loads. Please respect the intended use (see Chapter 2.5)!
- When carrying out setting or adjustment work on the wheelchair there is a risk that body parts could get pinched!
- Keep items of clothing and body parts away from gaps between moving parts!
- When negotiating obstacles (steps, etc.) call in an attendant. Use tipping aids (see Chapter 4.10).
- Only go up or down steps or large obstacles with the help of at least two attendants (see Chapter 6.3). Use equipment such as ramps, lifts or stair-climbing devices, if these are available.
- When driving on downhill/uphill slopes, please be aware that even at a few degrees, there is a risk of tipping over. We recommend the use of anti-tip wheels (see Chapter 4.1.1).
- Avoid collisions of any type!
- Only drive on level, firm ground.
- When transferring to or from the wheelchair, apply both parking brakes (see Chapter 4.5).

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

- When on roads, observe the road traffic regulations.
- Each time, before setting off, check the reflectors of your wheelchair to make sure they are not damaged and are visible for other road users.
- Only use the parking brakes when parking.
- The seating materials are frame resistant. Please keep cigarettes and other possible ignition sources away from your product.
- Observe the storage and operating conditions (see Chapter 7)
- When transferring between the wheelchair and the bed (see Chapter 6.1 and 6.2) always
 - apply both parking brakes
 - flip footplate up or swing the legrests away
- Notification of incidents. The provider or user must report
 1. Any malfunction
 2. Any change in the characteristics or performance as well as
 3. Any inadequacy in the labelling or instructions for use of a medical product, which has led or could lead to death or a severe deterioration in the health condition of a patient, an employee or a third party, must be reported immediately to your appropriate state authority responsible for medical products. They will pass the report on to the authority responsible for the provider immediately and will also inform the manufacturer and the authority responsible for the manufacturer.

Protect the wheelchair from sunlight to avoid burns caused by heated components

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

2.2. Risk of tipping over

The anti-tip wheels which are fitted on both sides (see Chapter 4.11) will largely prevent the wheelchair from tipping over backwards. However please note that driving with anti-tip wheels is only possible in a limited way on ground with different levels, lift platforms and ramps. Make sure there is sufficient gap at the top, bottom and sides.

2.3. Safety in vehicles

To see whether your wheelchair is permitted for use as a seat in a vehicle for transporting individuals with impaired mobility or not, you can refer to the symbols on the product's identification label (see Chapter 3.3):



Wheelchair is designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. A)

Wheelchair is not designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. B)

If your wheelchair is not designed to be used as a seat in a vehicle, under no circumstances must it be used as such. It does not fulfil the requirements in compliance with ISO 7176-19. Violation can cause the most serious injuries resulting in death in the case of an accident (Fig. B).

2.4. Use on roads

The wheelchair is designed for indoor and outdoor use. Please note that when you drive on public roads you must comply with road traffic regulations. Do not endanger other users through reckless driving, this applies particularly on pavements.

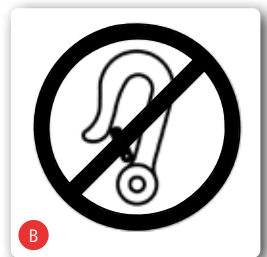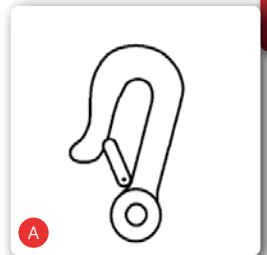S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

2.5. Intended use

The wheelchair is designed to increase the mobility only for those who have difficulty walking with the indications specified (see Chapter 2.6).

The maximum user weight for the:

S-Eco 2 of 125 kg

S-Eco 300 of 125 kg

S-Eco 300 XL of 170 kg

Pyro Start of 125 kg

must not be exceeded.

Please note:

We can only provide a warranty if the product is used under the conditions specified and for the intended purposes.

2.6. Indications

Provision of a wheelchair is intended for individuals who are unable to walk or who have severely impaired mobility, e.g. through:

- paralyzes
- loss of limbs
- limb defects/deformity
- joint contractions (not on both arms)
- joint damage (not on both arms)
- other illnesses

2.7. Contraindications

Provision of wheelchairs is not suitable for individuals who have:

- no capability to sit
- joint damage/contractions on both arms
- perceptual disorders
- severe equilibrium disorders
- loss of limbs on both arms
- reduced and insufficient vision
- severe limitations of cognitive abilities

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

2.8. Declaration of conformity

We, Bischoff & Bischoff GmbH, declare under sole responsibility, that the S-Eco 2 / S-Eco 300 / Pyro Start wheelchair meets all requirements of MDR 2017/745, which are applicable.

2.9. Liability

We can only provide a warranty if

- the product is used under the conditions specified and for the intended purpose.
- Modifications, enhancements, repairs and maintenance work is carried out only by those individuals who have been authorised by us to do so, and
- the wheelchair is used in accordance with the instructions for use.

2.10. Service life

The expected product service life is up to 5 years. Prerequisites for this are proper use as well as compliance with the safety, care and maintenance information in accordance with this user manual.

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

3. Product and scope of delivery

3.1. Checking the delivered goods

All products from Bischoff und Bischoff GmbH undergo an appropriate final inspection at our factory and the CE mark is affixed (see Chapter 3.3).

The wheelchair will be delivered in a special cardboard box. After unpacking, retain the cardboard box if possible. It can be used for possible later storage or return of the product.

Delivery and induction is usually carried out by a qualified medical equipment retailer.

When sending by train or shipping company, the goods must be checked immediately for transport damage in the presence of the forwarding agent.

Check that the delivery is complete and undamaged (see Chapter 3.2). In the case of anomalies or damage, contact our Customer service, Tel.: +49 (0)700/6000 7070.

3.2. Scope of delivery


On receiving the goods, please immediately check that the delivery is complete. The delivery consists of:

- packing,
- wheelchair (preassembled),
- legrests
- this user manual
- any necessary accessories

3.3. Product's identification label and serial number

The product's identification label (Fig. 01) and serial number are located on the cross brace.

A	Manufacturer's logo
B	Manufacturer information
C	MD Medical product
D	CE mark
E	Observe safety in vehicles
F	Attention! - read the User manual
G	Important! - read the User manual
H	Model name
I	Model number
J	Max. payload
K	Date of manufacture
L	UDI number
M	Serial number

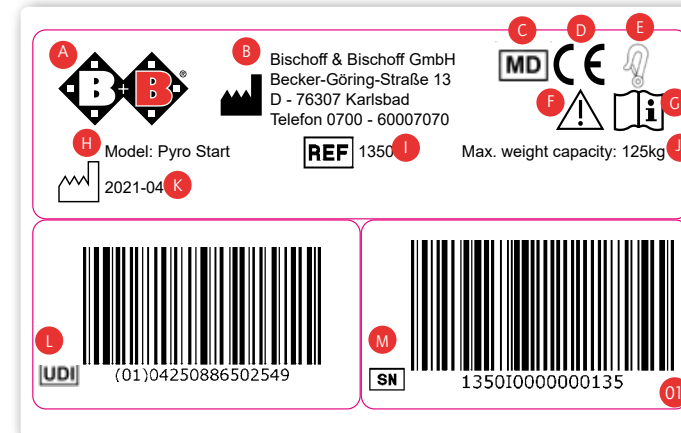 Depending on the model number, the information may vary. Refer to your product's identification label for the information which relates to your product.


S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

3.4. Overview

(see Fig. 2)

A	Push handle
B	Back cushion
C	Castor
D	Sideguard
E	Seat pad
F	Drive wheel
G	Quick-release axle
H	Parking brake
I	Handrims
J	Legrest with heel strap

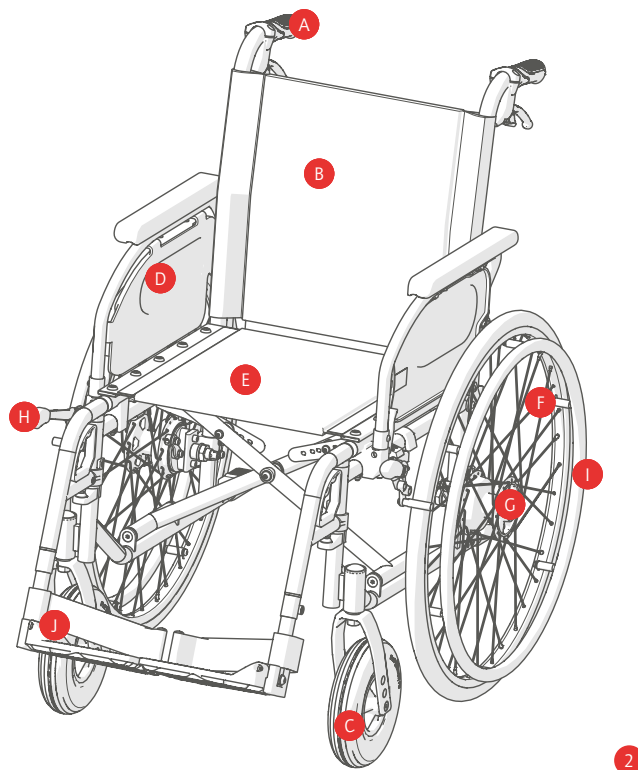4. Assembly/Adjustments

Your new B+B wheelchair will be delivered in a B+B original cardboard box, assembled and folded.

In order to avoid transport damage, detachable attachments are delivered separately with the product.

Refer to the technical data to see which options are possible on your wheelchair (see Chapter 7).



4.1. Folding and unfolding

Unfolding

- Position the folding wheelchair next to you. (Fig. 3)
- Tip the wheelchair towards you until the weight is off one of the drive wheels.
- Push the seat tubes apart and push them down with the flat of your hand, until they lock into place in the bracket on the frame (Fig. 4).
- The seat pad must be completely unfolded.
- Apply the left and right parking brakes on the drive wheels.
- Fit the legrests.
- You can now sit down


Folding

- Position the folding wheelchair next to you.
- Flip the footplates up (Fig. 5)
- Remove the seat cushion, take hold of the front and rear of the seat pad and pull it up,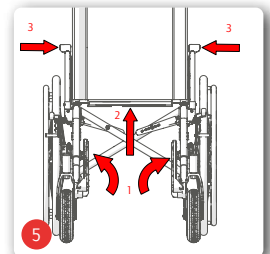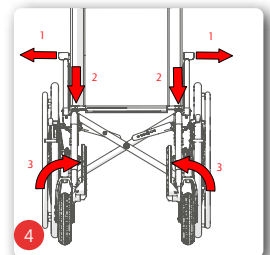

When folding and unfolding, watch your fingers as there is a risk of pinching

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

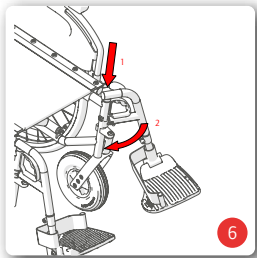
4.2. Legrests

4.2.1. Swing-away legrests

You can flip the footplates up separately to the side and adjust to the lower leg length of the user. You can remove the legrests, swing them away and refit them without needing to use tools.

Attach legrests


- Hold the top of the legrest so that it is pointing outwards.
- Guide the legrest into the hole in the frame using the guide bolts (Fig. 6).
- Swing the legrest to the front until you hear it clearly click into place (Fig. 6)
- Flip the footplate down if necessary.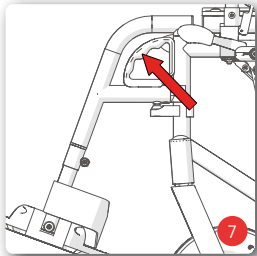

Swing the legrests away and remove them

In order to make it easier to transfer to and from your folding wheelchair, you can swing the legrests inwards or outwards, without having to remove them.

- Flip the footplate up if necessary.
- Pull the handle upwards (Fig. 7) and swing the legrest outwards (Fig. 8).
- In the outward position, the legrest can be pulled out by pulling it upwards (Fig. 8).

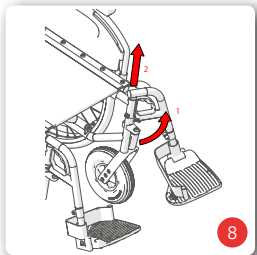Before setting off, fit the legrests as described above.

Never stand on the footplate and when swinging it away, watch your fingers as there is a risk of pinching

Standard wheelchair | **S-Eco 2 | S-Eco 300**
 Special size | **S-Eco 300 XL**
 Lightweight wheelchair | **Pyro Start**

4.2.2. Adjusting the legrests

Loosen the screw on the clamp mounting, then adjust the legrest to the required length and tighten the screw (Fig. 9). Please note the grooved marking for the maximum adjustment.


4.2.3. Horizontally-adjustable legrests (optional)

Assemble, disassemble and swing them away as for the standard legrests.

To adjust, undo the clamp lever, then adjust the legrest to the required length and then tighten the clamp lever

The horizontal adjustment works as follows:

- Push the locking lever forwards.
- by lifting or lowering the footplate receiver tube you can get the legrest into the required position.
- Release locking lever; legrest locks into place automatically.

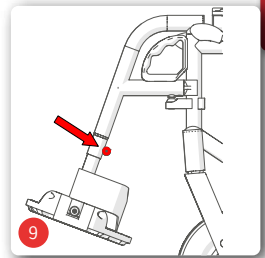Never stand on the footplate and when swinging it away, watch your fingers as there is a risk of pinching

To change seat, the legrest must be swung away or removed. Never sit on the legrest. Risk of tipping over!

4.3. Amputee legrests (optional)


Assemble, disassemble and swing them away as for the standard legrests, adjustment

and the horizontal adjustment works as for the horizontally-adjustable legrests:

To change seat, the legrest must be swung away or removed. Never sit on the legrest. Risk of tipping over!


S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

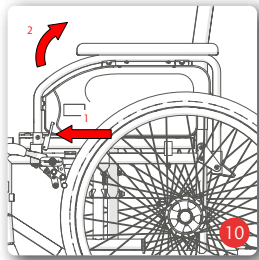4.4. Sideguards

4.4.1. Universal sideguard

- In order to be able to transfer more easily, both sideguards with armrests can be swung backwards.
- Pull the locking lever gently and at the same time, swing the sideguard to the back (Fig. 10).
- To lock the sideguard, flip it forwards and click it into place on the armrest using gentle pressure.
- The arm pads on the sideguards are fitted in the central position when delivered.
- They can be fitted in either the front or rear positions.

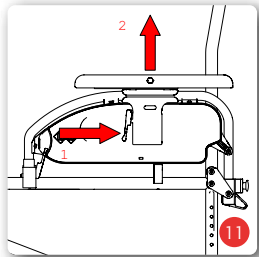When swinging the sideguards away, make sure that clothing or other items cannot be trapped. Before using for the first time, check that the sideguards are firmly in place!

Never use the sideguards to carry the wheelchair!

4.4.2. Height and depth-adjustable sideguards (optional)

As an option, your wheelchair can be fitted with height and depth-adjustable sideguards. The sideguard can be removed and swung away.

Height adjustment

- Pull the locking lever to move the armrest to the required position (Fig. 11).
- When you have reached the position required, then release the lever. The lock clicks into place.
- The height has been correctly selected if the upper arm and forearm are approximately at right angles to one another when the forearm is resting on the armrest.

Standard wheelchair | **S-Eco 2 | S-Eco 300**
 Special size | **S-Eco 300 XL**
 Lightweight wheelchair | **Pyro Start**

Depth adjustment

- Press the knob on the side of the armrest and push the armrest to the required position (Fig. 12).
- Release the knob and move the armrest gently forwards or backwards until it clicks into place.

4.5. Brakes



As standard, the wheelchair is fitted with parking brakes to be operated by the wheelchair user.

The parking brakes must not be used as operational brakes, as there is a risk of death due to the risk of falling!

Please note that there is a risk of burns if your hands heat up when braking using the handrims!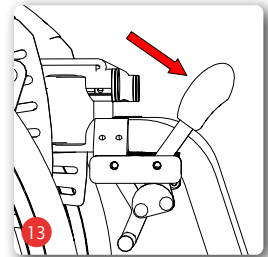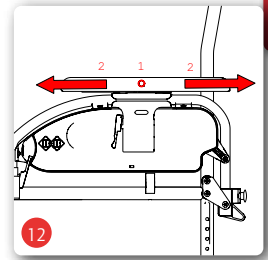


4.5.1. Operating the parking brake

The parking brake must only be used for parking. To slow down while driving, you must only use the drive wheel handrims or the optional drum brakes for attendant.

- When braking using the handrims, hold the highest point. First let the handrims slide through your hands
- Slowly increase the braking force by exerting gripping force on the handrims, until you come to an abrupt stop.
- As soon as the wheelchair has stopped, push both brake levers forward (Fig. 13). The wheelchair is securely braked when it can no longer be pushed and both brake levers are pushed forward to the stop position.
- Pull the lever towards you to release the brake.

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

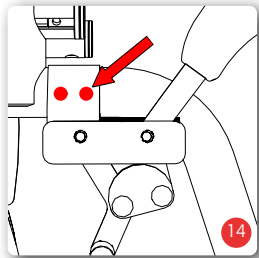Never use the brake lever as a support - e.g. when transferring or standing up -, otherwise there is a risk of tipping over and falling! The lever could break off!

The parking brake - with or without brake lever extension - only works on one drive wheel and must therefore always be used on both sides!

4.5.2. Adjusting the parking brakes

Adjusting the brakes must only be carried out by trained personnel. Please bear in mind that if the brakes are set incorrectly this can have life-threatening consequences.

- Move the braking system to a position that offers a safe braking effect in all driving situations and can be actuated.
- To adjust the gap, undo the two socket screws on the brake bracket (Fig. 14).
- Then push the brake system to the correct position along the bracket.
- Finally re-tighten the two socket screws.
- Make sure that the brakes click into place when operating the lever and that they don't snap back.

Standard wheelchair | **S-Eco 2 | S-Eco 300**
 Special size | **S-Eco 300 XL**
 Lightweight wheelchair | **Pyro Start**

4.5.3. Operating the parking brake with the brake lever extension

The procedure for operating the parking brake with the brake lever extension is the same as operating the parking brake (see Chapter 4.5.1).

Folding the brake lever extension in




- The brake lever can be folded in when driving or transferring to the side.
- Pull the plastic grip out of the lever (Fig. 15).
- Fold the plastic grip down (Fig. 16).
- Before setting off, the brake must be returned to its operational position by following the procedure in reverse order.

Never use the brake lever as a support - e.g. when transferring or standing up -, otherwise there is a risk of tipping over and falling! The lever could break off!

The parking brake - with or without brake lever extension - only works on one drive wheel and must therefore always be used on both sides!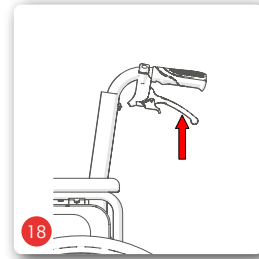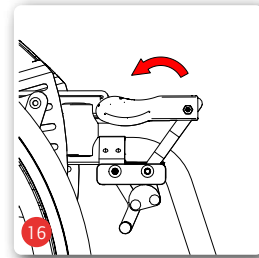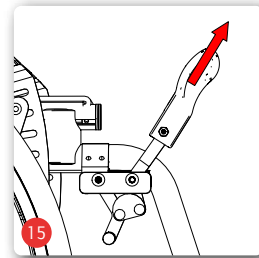


4.6. One-hand brake (optional)

Using the one-hand brake, you have a parking brake which you can apply to both drive wheels using a brake lever.

4.6.1. Drum brake (optional)

As an option, the wheelchair can be fitted with a drum brake which can only be operated by an attendant using the brake lever on the push handles.


S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

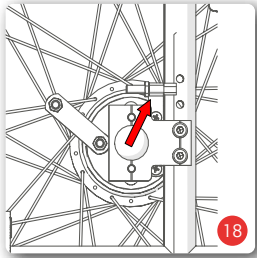Adjusting the drum brake

Basic setup for the drum brake is carried out on the cable clamping screw. Fine adjustment is carried out using the adjusting nut on the brake cable. Unscrewing the adjustment nut will increase the braking effect (Fig. 18).

- Drum brakes only work on one drive wheel at a time!
- Drum brakes are also subject to natural wear. If the braking effect decreases, the wheelchair must no longer be used. In order to avoid this happening, for your own safety you should check that the brakes are working properly each time before setting off!
- Once you have finished the adjustment, re-tighten all screws and carry out a brake test!

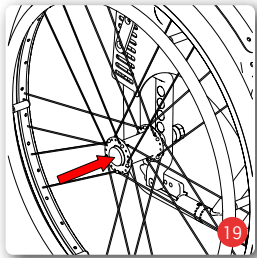Adjusting the brakes must only be carried out by trained personnel. Please bear in mind that if the brakes are set incorrectly this can have life-threatening consequences.

4.7. Wheels and tyres

The seat height can be adjusted to 3 positions to meet the user's requirements (except S-Eco2).

Drive wheel and castor must always be located in the same position respectively to each other

. UPPER - MIDDLE - LOWER

If the drive wheel and castor are not at the same position, there is a risk of tipping over and falling.

4.7.1. Quick-release axle

As standard the wheelchair has quick-release axles. This means you can easily remove and refit the drive wheels.

- Push the lock of the quick-release axle using your thumbs and pull the wheel off (Fig. 19).

Standard wheelchair | **S-Eco 2 | S-Eco 300**
 Special size | **S-Eco 300 XL**
 Lightweight wheelchair | **Pyro Start**

- When refitting, simply put the wheel onto the receiver. When doing this, push the locking knob in. The click you will hear from the quick-release axle will indicate that the wheel is securely mounted. After fitting, make sure that the wheels are tight!

4.7.2. Drive wheels

- Remove the drive wheels using the quick-release axles.
- Undo the screws on the adapter and pull them off completely (Fig. 20).
- Position the adapter to the required seat height.
- Insert the screws back into the adapter and re-tighten the screws.

If your wheelchair has the optional pneumatic tyres, pump the wheel to the pressure according to the value on the tyre's wall.



Please contact your dealer in case of puncture in your pneumatic tyres.

4.7.3. Castors

- Undo the axle and remove it completely.
- Position the castor in the position for the required seat height (Fig. 21).
- Put the axle back into the castor fork and tighten

If the drive wheel or castor is worn or defective, ask your dealer to service them!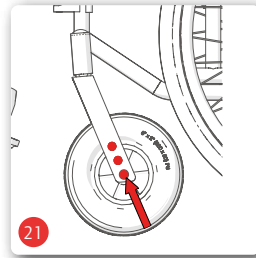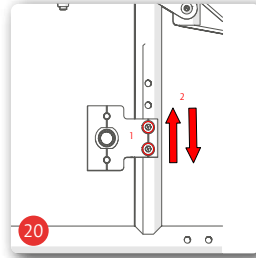

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Position the castor in the same hole position as the drive axle. This is the only way to guarantee good directional stability for the wheelchair.

4.8. Transit wheels (optional)

With the transit wheels you can reduce the width and length of your wheelchair. Please note that you no longer have handrims. You can only retrofit the transit wheels when the chair is unoccupied.

4.9. Wheelbase extension (optional)

The wheelbase extension improves the stability and therefore reduces the risk of tipping over backwards. Negotiating obstacles and tipping the wheelchair becomes more difficult though.

4.10. Tipping aids

In order to make it easier for an attendant to negotiate obstacles such as kerb edges, tipping aids can be used. Hold the wheelchair firmly with both hands on the push handles. Press one foot onto one of the tipping aids and at the same time push the hands down (Fig. 22).

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

4.11. Anti-tip wheels (optional)

The anti-tip wheels prevent the wheelchair from tipping over backwards. There must be a gap between the rollers of the anti-tip wheels and the ground of approx. 30-50 mm (Fig. 23).


Anti-tip wheels are particularly recommended for inexperienced wheelchair users or amputees.

For safety reasons, always use the anti-tip wheels as a pair!

For adjustment, contact your dealer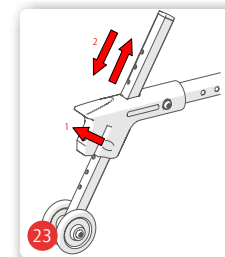

4.12. Headrest (optional)

The headrest is mounted on the push handles. After undoing the clamp lever or the star screw you can adjust the headrest to the right position. Then re-tighten the screws.

4.13. Adjustable backrest (optional)

- Flip the cover up.
- Adjust the length of the straps, in order to adjust the backrest slack.

4.14. One-hand operation (optional)

With one-hand operation you have both handrims on one wheel, so that you can operate both drive wheels with one hand. The option can be fitted either left or right.

4.15. Reflectors

As standard your wheelchair is fitted with reflectors on the back and spokes. These must not be covered.

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

5. Accessories

5.1. Lap belt (optional)

You can order a lap belt as an option and this will secure users who do not have the required stability in the wheelchair.

- Remove the sideguard or swing it away (see Chapter 4.4).
- Wrap the open end of the belt once round the sideframe tube
- Then fix the belt strap with the buckle.
- You can adjust the belt length using the sliding tabs on the belt.
- Use the fastening clip to open and close the belt.

5.2. Therapy table (optional)

The table is simply pushed into the armrest from the front. To remove the therapy table, simply pull it back out.

5.3. Togo (optional)

An electrical Togo pushing & braking aid can be fitted to your wheelchair.

If required, see the Togo pushing & braking aid user manual for all relevant information on assembling and use.

5.4. Crutch holder (optional)

Put the crutch in the crutch holder and then fix it to the backrest
Never stand on the crutch holder and never use it as a tipping aid.

5.5. Spokeguard (optional)

The spokeguard is fixed to the spokes using the cable ties.

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

5.6. Handrim cover (optional)

Put the handrim cover over the handrims in order to achieve better grip on the handrim, for example if your hand strength is limited.

5.7. Infusion & oxygen bottle holder (optional)

The infusion & oxygen bottle holder is fixed on the back tube of your wheelchair.

Attention - with increased load from the oxygen bottles, the risk of your wheelchair tipping over backwards will also increase. We strongly recommend the use of anti-tip wheels.

5.8. Fixing a bag (optional)

If you want to fit a bag to your wheelchair, hang the straps of your bag over the back tubes of your wheelchair.

Please note that the reflectors must not be covered.

Attention - with increased load in the bag, the risk of your wheelchair tipping over backwards will also increase. We strongly recommend the use of anti-tip wheels.

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

6. Use

6.1. Side transfer

Never stand on the footplate and when swinging it away, there is a Risk of tipping over!

- Make sure that the sides of the wheelchair and the current or future seat surface are as close to each other as possible.
- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- Secure the wheelchair using the parking brakes (see Chapter 4.5.1).
- If necessary, swing the armrest backwards or remove the sideguard (see Chapter 4.4).
- Flip both footplates up, swing them to the side and remove them (see Chapter 4.2).
- Slide sideways onto the other seat surface. When doing this sit as far back as possible.
- Move the legrests back into their original position.
- If necessary lock the sideguards back in place.

6.2. Frontal transfer

Never stand on the footplate and when swinging it away, there is a Risk of tipping over!

- Remove the legrests (see Chapter 4.2).
- Make sure that the front of the wheelchair and the current or future seat are as close to each other as possible and apply the parking brakes on both sides (see Chapter 4.5.1).

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- Now slide onto the seat surface by rotating your body.
- Then hang the legrests back into the initial position and flip the footplates down.

6.3. Going up or down stairs or a high obstacle


Only the frame and the push handles must be used as grip points.


The helpers only grip on the firmly fixed frame parts (Fig. 24). In order to go up stairs, the rear helper pulls the wheelchair backwards up the steps by holding onto firmly-mounted handles. The lower helper grips on the front frame tubes and stabilises the wheelchair. When doing this he pushes the drive wheels of the wheelchair onto the steps. When descending, the lower helper brakes the chair by pressing the wheelchair straight into the steps. The upper helper holds the wheelchair by the push handles, keeps it secure and holds it in the right position. The wheelchair should roll down step by step.

Two attendants must help with obstacles and stairs with more than three steps.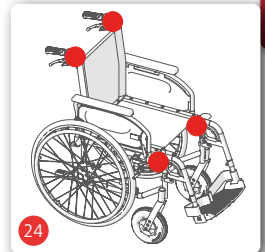

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

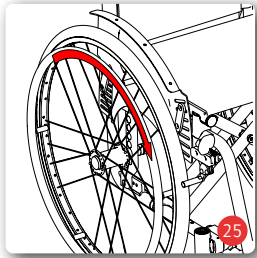6.4. Driving the wheelchair

- Sit in your B+B wheelchair. Make sure that you sit as far back as possible.
- Propulsion is achieved using the handrims on the drive wheels. Place your hands on the handrims using your thumbs and bent index fingers.
- From this position, push the wheelchair with both hands (Fig. 25).
- Turning right: hold the right handrim tight and only push with the left one
- Turning left: push in the opposite way
- Turning on the spot: push the handrims in opposite directions at the same time
- Braking: reduce the speed of the handrims with your hands.

Carry out the first test drives carefully, until you have got used to the wheelchair and its driving characteristics. Wheelchairs only have limited security against tipping over and slipping. Take extra care when braking, starting off or turning on slopes. Extended leaning out of the wheelchair also increases the risk of tipping over. Tipping protection reduces this risk. For easier obstacle negotiation, the attendant can tip the wheelchair (see Chapter 4.10).

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

7. Technical data


	S-ECO 2	S-ECO 300	Pyro Start	S-Eco 300 XL
Dimensions and weights				
Total length with legrests (mm):	1050	1010	1010	101
Total length without legrests (mm):	815	775	775	775
Total width (mm):	SW + 190			
Total width, folded (mm):	280	330		
Total width with drum brake (mm):	SW + 220			
Total width with drum brake, folded (mm):	310	360		
Total height (mm):	925	935	935	935
Push handle height (mm):	925	935	935	935
Lower leg length (mm):	400	400	400	400
Kerb weight (kg):	18.2	18.4	16.2	22,8
Max. payload (kg):	125	125	125	170
Default seat angle (°):	90			
Seat width (mm):	370, 400, 430, 460, 490, 520			520, 550, 580
Seat depth (mm):	420			450
Front seat height without seat cushion (mm):	550	470, 490, 510		
Rear seat height without seat cushion (mm):	470	440 - 490		
Backrest angle (°):	90			
Backrest height without seat cushion (mm):	400			
Armrest height without seat cushion (universal sideguard) (mm):	215			
Armrest height without seat cushion (height and depth-adjustable sideguard) (mm):	190 - 265 in 5 steps of 15 mm			
Armrest length (mm):	330			
Armrest width (mm):	40			
Turning area (mm):	1200			
Static stability (°):	7			

	S-ECO 2	S-ECO 300	Pyro Start	S-Eco 300 XL
Parking brake operating force (N)	< 60			
Tyres				
Drive wheels (inch):	24" PU			
Transit wheels	12" PU			
Castors (mm):	7" x 1.75" PU	8" x 2" PU	8" x 1.25" PU	8" x 2" PU
Material				
Frame:	Steel			
Cross brace, sideguard tube, legrest tube	Steel	Steel	Aluminium	Steel
Seat and backrest:	Nylon			
Storage conditions				
Temperature:	-40 - +65°C			
Humidity	45 - 60%			
Operating conditions				
Temperature:	-25 - +50°C			
Humidity	20 - 80%			
Accessories and options for all models				
Height-adjustable push handles, height-adjustable sideguard with combi armrest, drum brake (total width + 30 mm), brake lever extension, transit wheels, horizontally-adjustable legrest, amputee legrest, wheelbase extension, retrofit PU -> pneumatic tyres, one-hand brake, one-hand drive, Togo pushing & braking aid, crutch holder, lap belt, anti-tip wheels (total length increases), seat cushion, spokeguard, therapy table, handrim cover, infusion & oxygen bottle holder, Apino Citybag, Apino Shopper, Headrest				
Accessories and options for individual models				
Legrest extension			X	
The dimensions stated are subject to manufacturing tolerances of +/- 10 mm.				

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**


8. Transport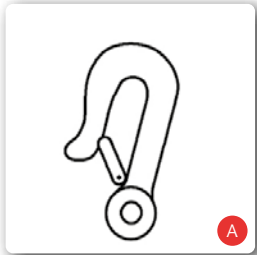

8.1. Transporting the wheelchair

In order to make the wheelchair as manageable as possible to transport, remove all removable parts (legrests, drive wheels, armrests, loose accessories) and fold the wheelchair up (see Chapter 4.1).

8.2. Transportation in vehicles

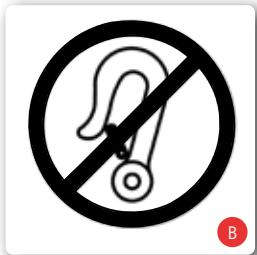To see whether your wheelchair is permitted for use as a seat in a vehicle for transporting individuals with impaired mobility or not, you can refer to the symbols on the product's identification label (see Chapter 3.3):

Wheelchair is designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. A)

Wheelchair is not designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. B)

All B+B products which have been approved as seating in vehicles have been tested in compliance with ISO 7176-19. Tests in compliance with ISO 7176-19 have been carried out with AMF-Bruns restraint systems. The correct fitting of the restraint system is a pre-requisite for safe transport of products in a vehicle for transporting individuals with impaired mobility and is the basis for the approval. Fixing points on the restraint system (Fig. A)

AMF-Bruns restraint systems provide load nodes (Fig. A) for fixing the products in the vehicle for transporting individuals with impaired mobility. The products must only be fixed to these load nodes for restraint in the vehicle for transporting individuals with impaired mobility. If possible, use a permanently installed car seat and its belt system. Stow the wheelchair in the boot. Those individuals responsible for the transportation should be familiar with the current legislation for transport as well as with the operation of the vehicle and its restraint system. Four-point restraint systems must be used to secure the wheelchair to the vehicle.

For any queries regarding how to secure your product in the vehicle for transporting individuals with impaired mobility, please contact B+B Customer Service on +49 (0)700/60007070.

If your wheelchair is not crash-tested, it must not be used as a vehicle seat under any circumstances. There is a risk of death!

9. Maintenance instructions for the user

9.1. Cleaning and disinfection

- Wipe the frame parts of the wheelchair with a damp cloth. If there is heavy contamination, also use a mild cleaner.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush).
- Wash the push handles, armpad, seat and back cushion with a mild cleaner.
- For disinfection use a disinfectant listed by the Association for Applied Hygiene.
- Upholstery parts can be machine washed at 40 ° C. Not machine dry, only drying at room temperature.

9.2. Checks before driving

9.2.1. Checking the brakes

Before setting off each time, check that the brakes work properly. With the parking brakes applied, the wheelchair cannot be pushed on dry surfaces. When the brakes are released, the wheelchair runs without scraping noises and good directional stability. If this is not the case, notify your dealer immediately and stop using the wheelchair until it has been serviced!

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

9.2.2. Checking the tyres

Each time before setting off, check the tyres for damage and if you have pneumatic tyres check that the air pressure is correct (see Chapter 4.7).

10. Repairs

Repairs and servicing work must only be carried out by authorised partners (medical supply stores or dealers). To find a partner in your area, please contact our Customer Service on tel.: +49 (0)700/6000 7070.

11. Maintenance information

Typical wear parts are brakes and tyres. Please contact your dealer for spare parts.

We recommend that you have your dealer check your wheelchair at least once a year.

If there are faults or defects, pass the wheelchair to your dealer for servicing immediately.

On request, any information and documents necessary for the repair and servicing will be provided to the dealer. If you have any questions, B+B Customer Service is available to help you on +49 (0)700/6000 7070.

The wheelchair must be checked in accordance with the maintenance plan and be in proper technical condition.

12. Passing the wheelchair on to another user

The wheelchair is designed for re-use. Before it is passed on to another user, it must be serviced by a dealer and hygienically cleaned. When passing the wheelchair on to another user please remember to hand over this user manual and the address of your dealer to the new user.

13. Storage/shipping

If the wheelchair is to be stored or shipped, all loose parts which are not fixed, except for the drive wheels, must be removed and packed individually in an appropriate cardboard box. The individually-packed parts can then be packed together in a larger cardboard box. We recommend that you keep the original packing and store it for these purposes, so that it is available when needed. Your wheelchair is then optimally protected against environmental influences during storage or transportation.

14. Disposal

If your wheelchair is no longer in use and needs to be disposed of, contact your dealer.

If you want to handle the disposal yourself, find out about the regulations in your area from local recycling companies.

S-Eco 2 | S-Eco 300 | Standard wheelchair
S-Eco 300 XL | Special size
Pyro Start | Lightweight wheelchair

15. Warranty

The warranty covers all defects of the wheelchair which are proven to be due to faulty material or manufacturing. In the event of a claim, submit the fully completed warranty certificate.

We provide the legal warranty of 2 years for our products.

For replacement devices, wear parts and repairs, the warranty period is 1 year.

Please follow the B+B care, warranty, hygiene and maintenance information. Your dealer can provide you with this.

Note!

Despite thorough research and editing, incorrect information may occur in our User manual. We reserve the right to make technical modifications. Images may show accessories. You can find the current version of the User manuals in the download area of our website:

www.bischoff-bischoff.com

Standard wheelchair | **S-Eco 2 | S-Eco 300**
Special size | **S-Eco 300 XL**
Lightweight wheelchair | **Pyro Start**

EN

Warranty certificate

Product: S-Eco 2 / S-Eco 300 / Pyro Start

Serial number:*

* (To be filled in by the dealer)

Dealer:

Date and stamp

*Signs are located on the cross brace

(Pay attention to our general terms and conditions.)

Failure to observe the instructions in the User manual as well as not carrying out maintenance work properly as well as, in particular, making technical modifications and additions (attachments) without the agreement of Bischoff & Bischoff GmbH will lead to loss of warranty and the manufacturer's liability.

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